# Politip

## Brugsanvisning Käyttöohjeet Bruksanvisning Productinformatie Οδηγίες Χρήσεως Kullanım Kılavuzu Инструкция Instrukcja stosowania

- Politip-F-polerer (grå) og Politip-P-polerer (grøn) til komposit og amalgam
- Politip-F-kiillottajat (harmaat) ja Politip-P-kiillottajat (vihreät) yhdistelmämuoville ja amalgamille
- Politip-F-polerere (grå) og Politip-P-polerere (grønn) for kompositmateriale og amalgam
- Polijstinstrumenten Politip-F (grijs) en Politip-P (groen) voor composiet en amalgam
- Σύστημα στίλβωσης Politip-F (γκρί) και Politip-P (πράσινο) για σύνθετα υλικά και αμάλγαμα
- Komposit ve amalgam için Politip-F cilalayıcılar (gri) ve Politip-P cilalayıcılar (yeşil)
- Полиры Politip-F (серый) и Politip-P (зеленый) для композитных материалов и амальгамы
- System gumek do polerowania wypiechli z materiałów kompozytowych i amalgamatu: Politip F - (szare, finiru-jące) i Politip P - (zielone, polerujące)

For dental use only.

Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed dentist.

Made in Germany  
Ivoclar Vivadent AG,  
FL-9494 Schaan/Liechtenstein

CE 0123

637200/1010/WE3/G

## Instructions for Use Gebrauchsinformation Mode d'emploi Istruzioni d'uso Instrucciones de uso Instruções de Uso Bruksanvisning

- Politip-F polishers (grey) and Politip-P polishers (green) for composite and amalgam
- Politip-F (grau) und Politip-P Polierer (grün) für Composite und Amalgam
- Pointes à polir Politip-F (grises) et Politip-P (vertes) pour les restaurations en composite ou en amalgame
- Strumenti di lucidatura Politip-F (grigi) e Politip-P (verdi) per composito ed amalgama
- Pulidores Politip-F (grises) y pulidores Politip-P (verdes) para composites y amalgamas
- Polidores Politip-F (cinza) e Polidores Politip-P (verde) para compostos e amalgamas.
- Politip-F-polerer (grå) og Politip-P-polerer (grøn) til komposit og amalgam

ivoclar  
vivadent®  
clinical

## English

### Instructions for Use

#### Description

Politip is an effective two-step polishing system for finishing and polishing dental composite and amalgam restorations. The polishers of the Politip system are available in the following three shapes: flame, small cup and large cup.

#### Composition

The polishers consist of silicone rubber, silicon carbide particles and titanium oxide.  
The shanks are made of stainless steel.

#### Indication

Politip is indicated for the finishing and polishing of restorations made of composite material and amalgam.

#### Contraindications

If a patient is known to be allergic to any of the above components, the polishers should not be used.

#### Side effects

None known to date.

#### Application

- **1<sup>st</sup> Step: Finishing with Politip F (grey)**  
With the Politip F finisher, excess material is removed and the restoration surfaces are smoothed.
- **2<sup>nd</sup> Step: Polishing with Politip P (green)**  
With Politip P the restoration can be polished to a high-gloss finish.

The restoration is finished and polished at medium contact pressure (approx. 2N). The finishing and polishing procedure is performed under water cooling, which also ensures evacuation of polishing residue.

If excess material has already been removed with fine-grit diamond grinding tools or the surface of the restoration has been finished with this type of instruments, the first step (Politip F) can be omitted.

When old restorations are polished, it is often sufficient to use Politip P only.

#### Politip F / Politip P:

- Recommended rotary speed: 7,500 - 10,000 rpm
- Only use with copious water spray (> 50ml/min)
- Use without polishing paste
- Sterilize polishers in the autoclave at 135 °C / 275 °F.

#### Proper use

- Use only straight/contra-angle handpieces which are technically and hygienically impeccable and regularly serviced and cleaned.
- The handpiece drives must run smoothly.
- The instruments must be correctly inserted and secured.
- Polishing should be performed with a slight rotary motion to avoid the creation of grooves.
- Jamming/leveraging should be avoided to reduce the risk of instrument fracture.
- Deformed instruments that do not run smoothly should cease to be used.
- From the point of view of safety, protective goggles and breathing protection should always be worn in order to avoid inhaling grinding dust. Efficient dust evacuation must be ensured during the polishing procedure.

#### Warning

**Improper use may lead to poor polishing results and a heightened health risk. This product must only be used by skilled persons.**

#### Rotary speed

The recommended and maximum admissible rotary speeds indicated in the Instructions for Use and on the packaging must be observed. They must not be exceeded. The polishing instruments tend to vibrate if the maximum admissible rotary speed is exceeded. This may lead to the destruction of the polishing instrument, deformation of the shank and/or fracture of the instrument and thus poses a potential danger to the user/practice team and the patient.

#### Warning

**Failure to observe the maximum admissible rotary speed results in a heightened safety risk.**

#### Contact pressure

High contact pressure during polishing will not produce an improved polishing result, but will lead to the increased generation of heat and accelerated wear of the polishing instrument. A mean contact pressure of approx. 2 N is sufficient.

#### Warning

**When polishing intraorally, excessive pressure needs to be avoided in order to prevent overheating and eventually pulp damage. In extreme cases, instrument fracture may result if too much pressure is applied, which can lead to injuries.**

#### Water cooling

In order to avoid unwanted heat generation on the tooth or soft tissue surface, sufficient water cooling (at least 50 ml/min) needs to be ensured. In addition, any residue produced during polishing is rinsed off by the water spray.

#### Warning

**Insufficient water cooling may lead to irreversible damage of the tooth, the restoration and the adjacent tissues.**

#### Storage, disinfection, cleaning and sterilization

- The polishing instruments should be stored in the original packaging at room temperature and protected from dust, moisture, pressure and re-contamination.
- Before using the polishing instruments on the patient for the first time, they must be disinfected and sterilized.
- After having used the polishing instruments in the mouth of the patient, they must be disinfected and cleaned and sterilized immediately.
- When manipulating contaminated instruments, protective clothing and gloves should be worn.
- Mechanically clean the instruments and immerse them in a disinfectant solution.
- Sterilize the polishers in the steam autoclave at 134 °C / 275 °F.
- The products are not suitable for sterilization in the chemi-clave or steam disinfectant.
- Care needs to be taken that the selected disinfectant is suitable for the disinfection of the polishers and that the instruments are disinfected correctly according to the instructions of the manufacturer. Please refer to the instructions of the

manufacturer for details on recommended use, immersion time and suitability of disinfection and cleaning agents. Aggressive substances, excessively long immersion times or incorrect dosage may damage the instruments and reduce the number of re-uses.

### Warning

**The suitability of the selected method for the disinfection/sterilization of the respective instrument must be verified in advance.**

### Recommendations to prolong the life of polishing instruments

Mechanical cleaning of the polishing instruments under running tap water and the subsequent sterilization in the autoclave increases the number of times the polishing instruments can be re-used. Alcohol-based disinfectant solutions may reduce reusability.

### Storage and shelf life

- Storage temperature 2 - 28 °C/36 - 82 °F
- Expiry date: See imprint on label
-  Sterilizable in the steam sterilizer (autoclave) at the temperature indicated
-  The respective instructions for use can be downloaded from the Ivoclar Vivadent website.

**Keep out of the reach of children!  
For use in dentistry only!**

**Date of preparation:** 9/2010; Rev. 0

### Manufacturer

Ivoclar Vivadent AG,  
FL-9494 Schaan/Liechtenstein

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions.

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**Ivoclar Vivadent Limited**
**Ivoclar Vivadent, Inc.**