Instructions for Use

Instructions for Use
Gebrauchsinformation
Mode d'emploi
Istruzioni d'uso
Instrucciones de uso
Instruções de Uso
Bruksanvisning
Brugsanvisning
Käyttöohjeet
Productinformatie
Οδηγίες Χρήσεως
Kullanım Kılavuzu

- Calcium hydroxide dressing
- Calciumhydroxid-Paste
- Pansement à l'hydroxyde de calcium
- Pasta all'idrossido di calcio
- Pasta de hidróxido de calcio
- Curativo de hidróxido de cálcio
- Kalciumhydroxid pasta
- Calciumhydroxid-pasta
- Kalsiumhydroksidipasta
- Kalsiumhydroksidpasta
- Calciumhydroxidepasta
- Φύραμα Υδροξειδίου του Ασβεστίου
- Kalsiyum hidroksit kaplama pati

For dental use only!

Caution: U.S. Federal Law restricts this device to sale by or on the order of a licensed dentist.

Made in Liechtenstein
Ivoclar Vivadent AG, FL-9494 Schaan

CE 0123

596610/0306/WE3/G

ivoclar
vivadent®
clinical

Description

ApexCal is a creamy, radiopaque, ready-to-use calcium hydroxide paste with a pH above 12.5.

Composition

ApexCal contains calcium hydroxide and bismuth carbonate in a mixture of water, glycerine, polyethylene glycol and other supplementary agents.

Indication

- Temporary disinfectant dressing, applied during endodontic therapy
- Indirect pulp capping or management of deep caries lesions
- Direct pulp capping

Contraindication

1. Direct pulp capping after exposure of the pulp in carious dentin, if:
 - the area of exposure is larger than 1 mm²,
 - a completely dry field cannot be established
 - the treatment of the tooth cannot be completed in one appointment
 - the tooth concerned is already causing discomfort.
2. Direct and indirect pulp capping if the pulp damage is irreversible
3. ApexCal should not be used in patients with a known allergy to any of the ingredients listed.

Side effects

There are no known side effects to date.

Interactions

ApexCal may reduce the detrimental effect exerted on the pulp-dentin complex by cements mixed with acids.

Application

Temporary disinfectant dressing

If ApexCal is used as a temporary disinfectant dressing, it is introduced directly into the prepared and dried root canal with the enclosed application tip or a lentulo spiral, and then spun up the canal to the apical region with the lentulo. Application may be repeated, if necessary. Tight sealing of the endodontic cavity using temporary filling material is essential in order to avoid the contamination of the root canal system which can occur between appointments.

Removal of ApexCal is accomplished by a suitable irrigation solution and instrumentation. The ApexCal paste needs to be removed before final obturation.

Direct and indirect pulp capping

With the help of the enclosed application tips, small amounts of paste are dispensed onto the mixing pad. Following this, ApexCal is applied to the exposed pulp or dentin near the pulp in thin layers and small patches using a suitable instrument. Subsequently, the surface is sealed with a layer of cavity lining material. It is important to observe contraindications!

Warnings

Contact of calcium hydroxide with the skin, eyes and mucous membrane may cause local irritation and burns. If inadvertent contact occurs, rinse with copious amounts of water and seek medical advice.

Storage

- Do not use ApexCal after the indicated date of expiration.
- ApexCal has to be stored at room temperature (12–28° C / 54–83 °F).
- After dispensing the material, remove the application tip from the syringe and re-seal the syringe with the original cap.
- Shelf life: See expiration date

Keep out of the reach of children.

For use in dentistry only.

Date of preparation: 05/2005

Manufacturer

Ivoclar Vivadent AG, FL-9494 Schaan
Principality of Liechtenstein

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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Ivoclar Vivadent AB
Ivoclar Vivadent UK Limited
Ivoclar Vivadent, Inc.