Silamat® S6

Form and function – perfectly mixed

Operating Instructions
Bedienungsanleitung
Mode d'emploi
Istruzioni d'uso
Instrucciones de uso
Instruções Operacionais

Page 3–6
Seite 7–10
Pages 11–14
Pagina 15–18
Página 19–22
Página 23–26

Bruksanvisning
Brugsanvisning
Käyttöohjeet
Bruksanvisning
Bedieningshandleiding
Οδηγίες Χρήσεως
Kullanım Talimatları

Sida 27–30
Side 31–34
Sivu 35–38
Side 39–42
Página 43–46
σελίδα 47–50
Sayfa 51–54

For dental use only!

Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed dentist.

Made in Austria

Ivoclar Vivadent AG, FL-9494 Schaan/Liechtenstein

ivoclar
vivadent
clinical

Operating Instructions


Safety

Intended use

Silamat S6 is a universal mixer for amalgam, glass ionomer cements and other predosed dental materials in capsules and syringes.

The apparatus has been developed solely for use in dentistry. Liability cannot be accepted for damage resulting from misuse or failure to observe the Operating Instructions. The user is responsible for testing the Silamat S6 for its use and suitability for the intended purposes. The intended use also includes the observation of the notes and regulations in these Operating Instructions.

The Silamat S6 has been shipped from the manufacturer in a safe and technically sound condition. The apparatus is built according to IEC 61010-1 (EN 61010-1) and EMV regulations. It complies with the relevant EU regulations and was certified to be technically safe by TÜV Product Service as internationally accredited testing body.

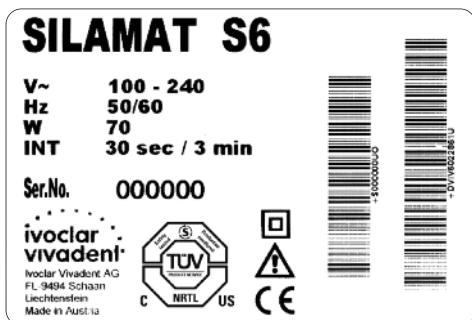Observe Operating Instructions

Do not use the appliance near flammable or combustible substances.

Double insulation (Apparatus complies with safety class II)

Switch the apparatus into stand-by mode.

Assumption of impaired safety

If it has to be assumed that safe operation is no longer possible, the power must be disconnected to avoid accidental operation. This may be the case, for example, if the apparatus is visibly damaged or no longer works correctly. A complete disconnection from the power supply is only ensured when the power cord is disconnected from the power source.

Operating voltage

Before you switch on the device, make sure that the voltage mentioned on the rating plate complies with your local power supply.

Location

Place the Silamat S6 in a place where optimum ventilation is ensured. The apparatus should neither be exposed to direct sunlight nor placed in the immediate vicinity of heaters. The Silamat S6 is only suitable for use indoors.

Dental materials used

The corresponding instructions for use of the material suppliers have to be observed.

If an amalgam capsule is leaking, the mercury has to be removed immediately. Mercury vapours represent an immediate health hazard. Pick up spillages of mercury with a reinforced paper (e.g. cardboard) or a plastic syringe and dispose in the hazardous waste in a closed container. Do not inhale vapours.

Operation

Start-up

1. Connect the power cord with the socket at the rear of the apparatus and subsequently with the mains socket.
2. Switch on the apparatus at the rear.

3. Raise the safety cover.

4. Place the mixing capsule in the mixing fork.

5. Close safety cover.
For safety reasons, the apparatus can only be operated if the safety cover is closed. If the safety cover is opened during the mixing process, the motor stops automatically.

6. Selecting the mixing time: The mixing time can be set between 1 and 30 seconds. In general a mixing time of 10 seconds can be used (unless other mixing times are recommended by the material supplier in the instructions for use). By default, the Silamat S6 was set to 10 seconds. The last used mixing time remains stored even if the apparatus had been switched off in the meantime.

7. Press "Start" key.
Once the set mixing time has been completed, the mixer stops automatically and the capsule can be removed.
If you wish to terminate the mixing process prematurely, open the safety cover.

Switch off apparatus

1. To switch off the apparatus, remove the power cord from the socket of the apparatus or the mains socket.
2. The switch on the rear side is only used to turn off the operability of the apparatus.

Cleaning

Switch off the apparatus by disconnecting the mains cord.
Clean housing, keypad and safety cover with a soft cloth and customary aldehyde-free disinfecting solution. Do not clean with highly aggressive

disinfecting solutions (e.g. solutions based on orange oil or with an ethanol content of more than 50%), solvents (e.g. acetone), or pointed instruments, which may damage or scratch the plastic. Make sure that no disinfectant, other liquid or foreign object enters the apparatus. The safety cover may be removed for cleaning by entirely opening the cover and carefully pulling it off to the back at the same time.

Warranty and repair

The warranty for the Silamat S6 is three years from the date of purchase. Malfunctions resulting from faulty material or manufacturing errors are repaired free of charge during the warranty period. For that purpose, the apparatus has to be sent C+F in the original packaging to the dealer or directly to Ivoclar Vivadent AG together with the corresponding receipt. Damages resulting from wear, transportation or incorrect start-up and improper use are not covered by the warranty.

Repair work may only be carried out by a certified Ivoclar Vivadent Service Center. Use only original spare parts.

Disposal

The apparatus must not be disposed in the normal domestic waste. Dispose of old apparatus according to the corresponding legal requirements in your country.

Product specifications

Delivery form
Art. No. 602 286
Silamat S6
Power cord
Operating Instructions

Technical data	
Mixing frequency:	approx. 4500 rpm, approx. 74 Hz
Operating voltage:	100 – 240 V 50/60 Hz
Rated capacity:	70 W
Overvoltage category II	
Pollution degree 2	
Operating mode:	max. 30 s on, 3 min. pause
Operating conditions:	
Temperature:	+10 °C to +40 °C
Relative humidity:	30% to 75%
Ambient pressure:	800 hPa to 1060 hPa (max. 2000 m)
Dimensions:	LxWxH = 240x230x170 mm
Weight:	3.2 kg
Transportation and storage conditions:	
Temperature:	-20 °C to +70 °C
Relative humidity:	10% to 75%
Ambient pressure:	500 hPa to 1060 hPa

This apparatus has been developed solely for use in dentistry. Start-up and operation should be carried out strictly according to the Operating Instructions. Liability cannot be accepted for damages resulting from misuse or failure to observe the Instructions. The user is solely responsible for testing the apparatus for its suitability for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes.

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Ivoclar Vivadent Limited
Ivoclar Vivadent, Inc.