Description

IPS Empress® Direct Color is a light-curing characterization material for the application of individualized characteristics in direct and indirect composite resin restorations and in CAD/CAM veneers fabricated chairside. Furthermore, it is suitable for masking discoloured tooth structure. IPS Empress Direct Color is applied below or between the composite resin layers.

Shades

IPS Empress Direct Color is available in the following shades:

Shades	Possible applications
White	<ul style="list-style-type: none"> • Imitating enamel stains • Simulating halo effects • Accentuating cusps and ridges
Honey Ochre	<ul style="list-style-type: none"> • Emphasizing the dentin core • Highlighting fissures • Imitating individual enamel characteristics
Blue Violet Grey	<ul style="list-style-type: none"> • Simulating translucent enamel areas • Imitating opalescent areas
Brown	<ul style="list-style-type: none"> • Imitating discoloured fissures, pits and enamel cracks

Composition

The monomer matrix (76–89.5 wt%) is composed of bis-GMA, urethane dimethacrylate and triethylene glycol dimethacrylate. The filler material consists of highly dispersed, silanized silicon dioxide (9–15.5 wt%). Additional components include stabilizers, initiators and pigments (< 9 wt%).

Indications

- Individualized characterization of direct and lab-fabricated composite resin restorations
- Masking of discoloured tooth structure
- Characterization of chairside CAD/CAM veneers

Contraindications

The material should not be used

- if the patient is known to be allergic to any of the components of IPS Empress Direct Color.
- if an adequately dry working field cannot be established or if the stipulated working technique cannot be applied.
- directly on dentin without dentin protection/dentin bonding agent.

Side effects

In rare cases, components of IPS Empress Direct Color may lead to sensitization. In such cases, the material should not be used.

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Interactions

Phenolic substances, e.g. containing eugenol/clove oil, inhibit the polymerization of methacrylate-based materials. Therefore, materials containing these substances must not be used in conjunction with IPS Empress Direct Color. Discolouration may occur in combination with cationic mouthwashes, plaque disclosing agents and chlorhexidine.

Application

IPS Empress Direct Color is used together with composite resin filling materials. In addition, it is used to characterize CAD/CAM veneers fabricated chairside (IPS Empress CAD), according to a special technique. Conditioning is carried out according to the instructions for use of the corresponding restorative/luting composite. IPS Empress Direct Color is dispensed from the syringe and applied with a brush or a probe. Furthermore, the syringes are supplied with thin, prebent metal cannulas with which IPS Empress Direct Color can be placed directly on the restorative material. In order to apply very small portions of the material, it is advisable to wipe the cannula with a cellulose pad from time to time to remove any excess material. The shade intensity can be reduced and the consistency thinned by mixing in Heliobond. Rubber dam isolation (e.g. OptraDam® Plus) is recommended.

Preparatory steps

1. Remove the syringe cap by twisting it until it snaps and then pulling it off the syringe.
2. Attach the applicator tip (Fig. 1) and twist it until it is flush with the syringe (Fig. 2).

A) Characterization of fillings

1. The composite resin (e.g. IPS Empress Direct, Tetric EvoCeram) is applied according to the directions of the manufacturer.
2. If IPS Empress Direct Color is used on large patches, it can be applied during the build-up of the restoration and then covered with the restorative material. In order to simulate enamel cracks and discoloured fissures, a notch is carved in the appropriate place in the last composite increment ("enamel" layer). The notch is filled with IPS Empress Direct Color using either the syringe tip or any other suitable instrument. Composite material is used to cover the notch starting from the sides. As a result, the characterization material is placed within the last composite layer and is therefore protected from wear.

Alternatively, IPS Empress Direct Color is applied after the filling has been built up:

A "characterization pattern" is ground in the filling material. Next, the pattern is etched with 37% phosphoric acid gel (e.g. Total Etch) for 15 s. The gel is subsequently rinsed off with water and the pattern is dried with air free of oil and moisture. In order to establish a sound bond with the polymerized restorative, the cleaned surfaces should be coated with Heliobond. Subsequently, IPS Empress Direct Color is applied with a brush or any other suitable instrument and polymerized according to the exposure time and the light intensity defined in Table 1.

The maximum layer thickness is 0.5 mm for all shades except Brown. The maximum thickness for Brown is 0.1 mm. Finally, the characterization material is covered with composite resin (e.g. IPS Empress Direct, Tetric EvoCeram) and polymerized according to the directions of the manufacturer.

B) Characterization of labfabricated composite restorations

In the build-up of inlays and onlays, IPS Empress Direct Color is used according to the procedure described for direct restorations (Section A).

C) Characterization of CAD/CAM restorations fabricated chairside

1. Try-in and subsequently adjust the veneers.
2. The surfaces of the ceramic veneers are cleaned and conditioned according to the directions of the manufacturer.
3. A thin coating of Heliobond is applied on the conditioned inner surfaces of the veneers and dispersed to a very thin film with blown air.
4. An extremely small amount of IPS Empress Direct Color is applied in a thin layer according to the individual requirements. It is important to note that excessively thick shade layers will compromise the fit and appearance of the restoration. Adjustments with rotating instruments as well as additional try-ins after characterization must be avoided, as the bond to the luting material is no longer ensured.
5. Heliobond and the staining materials are polymerized according to Table 1.
6. The veneers are placed according to the directions of the manufacturer of the luting composite.

Special instructions

1. Only the cannulas designed for the application of IPS Empress Direct Color should be used. Other types of cannulas may not fit the syringe properly and may become loose during the application procedure.
2. IPS Empress Direct Color is best used when it has attained ambient temperature. At refrigerator temperatures, the contents of the syringe may be difficult to dispense.
3. IPS Empress Direct Color must not be exposed to bright light during its application.
4. In order to ensure long-lasting shade effects, IPS Empress Direct Color must not be applied as the final layer on the surface of the restoration. If possible it should be placed as an intermediate layer or in carved notches.
5.  If IPS Empress Direct Color is applied directly in the mouth of the patient, the application cannula of the syringe must only be used for one patient due to hygienic reasons (prevention of cross-contamination between patients).
6. Oxidizing disinfectants must not be used to disinfect syringes.

Warning

Contact of uncured IPS Empress Direct Color with skin, mucous membrane and eyes must be avoided. Unpolymerized IPS Empress Direct Color may have a slight irritating effect and lead to a sensitization to methacrylates. Commercial medical gloves do not provide protection against the sensitizing effect of methacrylates.

Storage and hygiene instructions

- Storage temperature: 2–28 °C (36–82 °F).
- IPS Empress Direct Color must not be used after the indicated date of expiration.
- Shelf life: See date of expiration on the syringe label or packaging.
- Syringes must be sealed immediately after use. Exposure to light may lead to premature polymerization.

Keep material out of children's reach.

For use in dentistry only.

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions.

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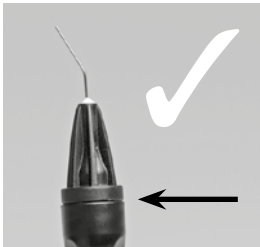
Fig. 2

Table 1

Unit	Bluephase C8	Bluephase Style	Bluephase	Bluephase 20i
Turbo	---	---	---	5 s
High Power	30 s	20 s	20 s	20 s
Soft Start	40 s	---	30 s	30 s

Other polymerization units

Exposure time	All shades except brown	brown
Lith intensity		
Increment thickness	up to 0.5 mm	up to 0.1 mm
Light intensity $\geq 500 \text{ mW/cm}^2$	40 s	40 s
Light intensity $\geq 1000 \text{ mW/cm}^2$	20 s	20 s

Ivoclar Vivadent AG

Bendererstrasse 2, 9494 Schaan, Liechtenstein

Tel. +423 235 35 35, Fax +423 235 33 60, www.ivoclarvivadent.com

Ivoclar Vivadent Pty. Ltd.

1 – 5 Overseas Drive, P.O. Box 367, Noble Park, Vic. 3174, Australia

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Tel. +1 905 670 8499, Fax +1 905 670 3102, www.ivoclarvivadent.us

Ivoclar Vivadent Shanghai Trading Co., Ltd.

2/F Building 1, 881 Wuding Road, Jing An District, 200040 Shanghai, China

Tel. +86 21 6032 1657, Fax +86 21 6176 0968, www.ivoclarvivadent.com

Ivoclar Vivadent Marketing Ltd.

Calle 134 No. 7-B-83, Of. 520, Bogotá, Colombia

Tel. +57 1 627 3399, Fax +57 1 633 1663, www.ivoclarvivadent.co

Ivoclar Vivadent SAS

B.P. 118, F-74410 Saint-Jorioz, France

Tel. +33 4 50 88 64 00, Fax +33 4 50 68 91 52, www.ivoclarvivadent.fr

Ivoclar Vivadent GmbH

Dr. Adolf-Schneider-Str. 2, D-73479 Ellwangen, Jagst, Germany

Tel. +49 7961 889 0, Fax +49 7961 6326, www.ivoclarvivadent.de

Wieland Dental + Technik GmbH & Co. KG

Schwenninger Strasse 13, D-75179 Pforzheim, Germany

Tel. +49 7231 3705 0, Fax +49 7231 3579 59, www.wieland-dental.com

Ivoclar Vivadent Marketing (India) Pvt. Ltd.

503/504 Raheja Plaza, 15 B Shah Industrial Estate

Veera Desai Road, Andheri (West), Mumbai, 400 053, India

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Tel. +31 23 529 3791, Fax +31 23 555 4504, www.ivoclarvivadent.com

Ivoclar Vivadent Ltd.

12 Omega St, Rosedale, PO Box 303011 North Harbour

Auckland 0751, New Zealand

Tel. +64 9 914 9999, Fax +64 9 914 9990, www.ivoclarvivadent.co.nz

Ivoclar Vivadent Polska Sp. z o.o.

Al. Jana Pawla II 78, 00-175 Warszawa, Poland

Tel. +48 22 635 5496, Fax +48 22 635 5469, www.ivoclarvivadent.pl

Ivoclar Vivadent Marketing Ltd.

Prospekt Andropova 18 korp. 6/, office 10-06, 115432 Moscow, Russia

Tel. +7 499 418 0300, Fax +7 499 418 0310, www.ivoclarvivadent.ru

Ivoclar Vivadent Marketing Ltd.

Qlaya Main St., Siricon Building No.14, 2nd Floor, Office No. 204

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Tel. +90 212 343 0802, Fax +90 212 343 0842, www.ivoclarvivadent.com

Ivoclar Vivadent Limited

Ground Floor Compass Building, Feldspar Close, Warrens Business Park

Enderby, Leicester LE19 4SE, United Kingdom

Tel. +44 116 284 7880, Fax +44 116 284 7881, www.ivoclarvivadent.co.uk

Ivoclar Vivadent, Inc.

175 Pineview Drive, Amherst, N.Y. 14228, USA

Tel. +1 800 533 6825, Fax +1 716 691 2285, www.ivoclarvivadent.us

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Ivoclar Vivadent AB