bluephase®

Licence to cure

new

LED for every use

What if...	2
Was ist wenn...?	3
Que faire en cas de...?	4
Cosa fare se... ?	5
Mantenimiento y limpieza	6
E se ...?	7
Vad är fel när ...?	8
Hvad gør man, hvis ...?	9
Miten tulee toimia, jos ...?	10
Hva skjer når ...?	11
Wat te doen als ...?	12
Σε περίπτωση που ...	13
Turkish	14
Russian	15
Co robić, gdy...	16

CE 0123

For dental use only!

Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed dentist

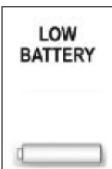
Made in Austria

Ivoclar Vivadent AG, FL-9494 Schaan / Liechtenstein

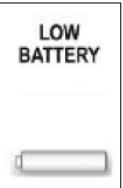
ivoclar
vivadent
clinical

What if...

What if....?

Symbol	Causes	Error Rectification
	Electronic defect in the handpiece or battery	Remove and reinsert the battery. If the error remains, replace the battery with the power pack (Click & Cure). If the error remains, please contact your dealer or your local Service Center.
	Apparatus is overheated or undercooled	Allow the apparatus to cool down (or assume room temperature if it is undercooled) and try again after a certain time. If the error remains, please contact your dealer or your local Service Center.
	Electronic defect in the battery	Remove and reinsert the battery. If the error remains, place the apparatus into the charging base. If the error remains nonetheless, replace the battery with the power pack (Click & Cure). Please contact your dealer or your local Service Center.
	Battery empty	Position the handpiece in the charging base. "Charging Battery" briefly appears in the display before the display returns to the stand-by mode. If the battery is not charged, the contacts have to be cleaned. If the error remains, please contact your dealer or your local Service Center.
No display during charging	Charging contacts contaminated Battery in safety mode (integrated protective circuit)	Check if the handpiece has been correctly placed into the charging base. Clean battery contacts. Information: "Charging Battery" briefly appears in the display before the display returns to the stand-by mode. If cleaning the battery contacts does not improve the results, remove the battery from the apparatus. Charge the battery separately from the handpiece in the charging base for approx. 10 minutes. If the error remains, please contact your dealer or your local Service Center.
LED of the charging base is not illuminated	Power pack not connected or defective	Check if the power pack is correctly positioned in the charging base (also check the charging contacts) or if the power pack is connected to the power supply by means of the power cord (display on the power pack lights up in green if it works correctly). If the error remains, please contact your dealer or your local Service Center.

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