Monobond-S

English

Instructions for Use

Mode d'emploi

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Instructions for Use

Description

Monobond-S is a single-component primer promoting an adhesive bond between luting composites (particularly the Variolink N and MultiLink N product lines) and indirect restorations made of glass-ceramics, lithium disilicate glass-ceramics, composites and fibre-reinforced composites).

Composition

Alcohol solution of silane methacrylate.

Indication

Monobond Plus serves as bonding agent and is used to create a durable chemical bond between luting composites and glass-ceramic, lithium disilicate glass-ceramic, composite and fibre-reinforced composite restorations.

Contraindications

- Any use not listed in the indications.
- If a patient is known to be allergic to any of the ingredients of Monobond-S.

Side effects

To date, no systemic side effects are known.

In rare cases, components of Monobond-S may lead to sensitization. Monobond-S should not be used in such cases.

Interactions

Contamination with saliva, blood or water must absolutely be avoided during the entire procedure.

Application

- 1 Adhesive cementation of prosthetic restorations:**
 - 1.1 Preparing the restoration
 - 1.1.1 Glass-ceramic restorations (e.g. IPS Empress®)
 - Etch with 5% hydrofluoric acid (e.g. IPS® Ceramic Etching Gel) for 60 seconds or according to the

For dental use only.

Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed dentist.

Made in Liechtenstein
Ivoclar Vivadent AG, FL-9494 Schaan/Liechtenstein

CE 0123

562132/1209/6spr/G

ivoclar
vivadent®
clinical

instructions of the manufacturer of the restorative material.

- Thoroughly rinse with water spray and dry with oil-free air.
- 1.1.2 Lithium disilicate glass-ceramic restorations (e.g. IPS e.max® Press, IPS e.max CAD)
- Etch with 5% hydrofluoric acid (e.g. IPS® Ceramic Etching Gel) for 20 seconds or according to the instructions of the manufacturer of the restorative material.
 - Thoroughly rinse with water spray and dry with oil-free air.
- 1.1.3 Composite or fibre-reinforced composite restorations
- Sandblast the restoration surface (according to the instructions of the manufacturer of the restorative material).
 - If necessary, clean the restoration in an ultrasonic unit for about 1 minute.
 - Thoroughly rinse with water spray and dry with oil-free air.
- 1.2 Apply Monobond-S with a brush or microbrush to the pre-treated surfaces. Allow the material to react for 60 seconds. Disperse with a strong stream of air. **IMPORTANT!** Contamination with saliva, blood or water must absolutely be avoided during the entire procedure. In case of contamination, the surfaces of glass-ceramic, lithium disilicate glass-ceramic and composite restorations can be cleaned with phosphoric acid. Subsequently Monobond-S must be applied again.
- 1.3 Apply the luting composite (e.g. Variolink N or Multi-link N) according to the respective Instructions for Use and insert the restoration.

Special notes

- If the primer dries out on the surface of the restoration before blasting with a strong stream of air, the surface needs to be rinsed with pure alcohol and blown with air.
- Do not insert the brush into the Monobond Plus bottle. This may lead to contamination.
- Carefully close the Monobond-S bottle immediately after use.

Warning

- Flammable.
- The product contains ethanol. Keep away from ignition sources.
- Monobond-S is an irritant. Avoid contact with the skin and mucous membrane.
If the material should accidentally come in contact with the skin, immediately wash with copious amounts of water.
- In rare cases, contact with the skin may lead to sensitization towards the ingredients.
- Commercial medical gloves do not provide protection against the sensitizing effect of methacrylates.

Storage

- Monobond-S should never be used after the expiration date.
- Storage temperature: 2–28 °C / 36–82 °F.
- Shelf life: see expiry date on label.

Keep out of the reach of children. For use in dentistry only.

Date information prepared

12/2009, Rev. 0

Manufacturer

Ivoclar Vivadent AG
FL-9494 Schaan / Liechtenstein

This material has been developed solely for use in dentistry. Processing should be done strictly according to the Instructions for Use. Liability cannot be accepted for damage resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

The Material Safety Data Sheet is available online at
www.ivoclarvivadent.com

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Ivoclar Vivadent Limited
Ivoclar Vivadent, Inc.