# Gnathometer M

Bedienungsanleitung  
Operating Instructions  
Mode d'emploi  
Istruzioni d'uso  
Instrucciones de uso

CE

ivoclar<sup>®</sup>  
vivadent<sup>®</sup>  
*technical*

# Gnathometer M

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# Gnathometer M

according to Prof. Dr. Böttger  
incl. UTS adapter

## Operating Instructions

### Introduction

The most reliable bite registration for edentulous cases is achieved by using the needle-point tracing technique.

When using Ivoclar Vivadent method of bite registration, however, there is no need for an additional appointment with the patient for registration purposes.

This procedure is part of the Ivoclar Vivadent Prosthetic System (BPS) and offers considerable advantages.

### Signs and symbols

Important information

Risks and dangers


Contraindication

### Range of application

- The Gnathometer M can be adapted to all edentulous cases without further modification.
- Newly, the Gnathometer M offers the possibility to connect the UTS transferbow directly. Skull/joint related model orientation is thus possible.
- From the preliminary impression to the finished denture, Ivoclar Vivadent offers dental technicians a scientifically based and practically proven system with state-of-the-art materials and apparatus.

### Parts List

- 1 Mounting jig
- 2 Upper registration plate with fixation blank
- 3 Lower registration plate with registration pin
- 4 Occlusal rims
- 5 Upper basic arch with UTS adapter
- 6 Lower basic arch
- 7 Adapters

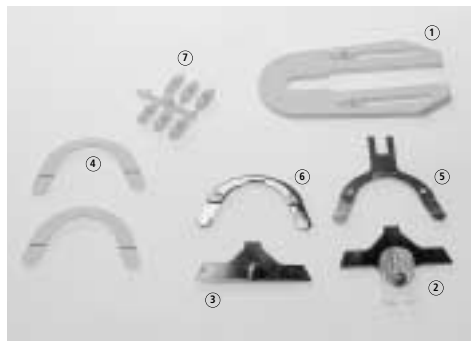### Technical and clinical procedure

#### In the laboratory

- Casting of the SR Ivotray double impression in the Biokop or Stratos, maintaining the individual inter-maxillary relationship  
If the occlusal localizing tray has been used, cast the impressions separately and orientate the models into the articulator/occludator by means of registration.
- Fabrication of individual trays together with the correct mounting of the Gnathometer M according to the intermaxillary relationship.

#### In the practice

- Once in the oral cavity, the occlusal rims of the Gnathometer M fit perfectly in the registration made with SR Ivotray or the occlusal localizing tray.
- Individual trays and registration apparatus form a unity. Together with the occlusal rims, they permit the free movement of the tongue necessary for impression taking. Subsequently, the occlusal rims can be replaced by the registration plates with a simple snap mechanism directly allowing needle-point tracing.
- After that, the transferbow may be either mounted according to the Camper's plane or Frankfort Horizontal of the patient and directly fixed to the Gnathometer M.

This system permits individual impressions, precise determination of inter-maxillary relationships, as well as skull-related transfer serving as a basis for further prosthetic works.

**Mounting the Gnathometer M in the laboratory**

By means of the mounting jig, the Gnathometer M is aligned as follows:

- dorsally to the retromolar pads;
- ventrally with the anterior rim over the bucco-labial fold (the result should be a vertical arch of anterior teeth);
- horizontally in the upper half of the ridge interspace; and
- bilaterally symmetrical to the midline of the palate

The Gnathometer M can be moved sagittally on the mounting jig. The registration pins should be placed in the high of the premolars. The mounting jig is bevelled dorsally and can be adjusted to the retromolar pads. Undercuts must be blocked out with wax.

Separate the models with Separating Fluid. After that, model the trays with SR Ivoclar or Spectra Tray in the usual manner.

Apply SR Ivoclar or Spectra Tray on the lower tray. Mount the Gnathometer M in the respective position in the unpolymerized material and fix.

Then the upper tray may be mounted. For this purpose, apply SR Ivoclar or Spectra Tray on the upper model and close the Biokop. This way, the Gnathometer M is mounted correctly and jaw-related.

Mounting jig and registration plate can be removed to facilitate finishing.

**Note**  
Ivoclar Vivadent recommends mounting the Gnathometer M directly on the tray by means of resin. This assures ample space and stability during the clinical procedure. It is mandatory to

check the correct position of the registration plates in the basic arches prior to the mounting procedure.

Instead of the registration plates, the white occlusal rims (4) can be used. The thickness of both rims corresponds with the thickness of the mounting jig and may therefore be closed tightly. The vertical relation remains unchanged.


**To sum it up, the following advantages can be achieved:**

- Personalized trays made of dimensionally stable material
- Correct mounting of the intra-oral registration device taking the spatial relation into account
- Functional impression, as well as intra- and extra-oral registration can be done in the practice during a single appointment.

### Clinical procedure

Check the personalized trays with occlusal rims for border moulding and, if necessary, make corrections. The occlusal rims close tightly and with even stress in the denture-supporting area.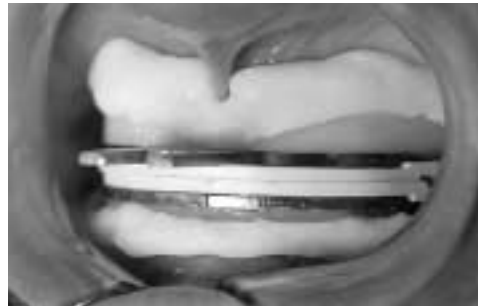

The antagonistic tray remains in the oral cavity during impression taking, assuring the correct relation for the subsequent registration.

### Registration

Remove occlusal rims and replace them by registration plates. Mount the registration pin to the mandible and the registration plate to the maxillary. Remove the fixation blank for the time being.


In order not to damage the border of the impression, position the plates with a pair of long-nose pliers. When the patient closes the mouth, only a single point contact will remain.

**Note:**

The patient should be instructed to place the tongue under the registration pin plate or dorsally in order to prevent possible injury by the registration pin when closing the mouth.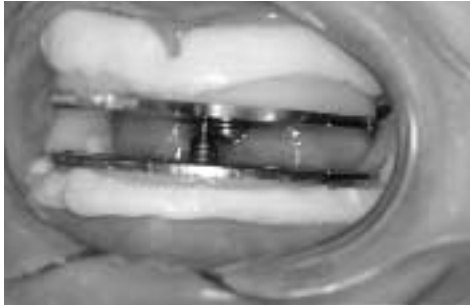

Check vertical relation. If necessary, make corrections by adjusting the registration screw. One full turn results in a vertical change of approx. 0.8 mm increasing to approx. 1.0 mm in the anterior region.

**Note:**

The screw thread leaves more possibilities to increase the vertical dimension than to reduce it. Therefore, Ivoclar Vivadent recommends taking the impression with SR Ivotray at a lower occlusal position and raising it to the correct position during actual registration.

A slight increase in vertical dimension during functional impression taking must also be considered.

After adjusting the vertical relation, the remaining space at the dorsal end of both impression trays must be checked with a flat instrument. The trays and the excess impression material must not touch each other. The freedom of movement of the mandible must not be impaired.

For needle-point tracing, the upper impression must be taken out of the mouth. The registration plate has to be removed, slightly warmed, and covered with a thin layer of wax colour (China Marker). After that, the plate has to be remounted.

Both functional impressions are in the mouth. Now the patient has to be instructed to effect the following movements:


It is recommended to practice these movements beforehand!

1. Move mandible forwards (protrusion) – backwards, repeat 1 x
2. Move mandible to the right (lateral movement) and back, repeat 1 x
3. Move mandible to the left (lateral movement) and back, repeat 1 x


**Important**

The patient must sit in a upright position without leaning his head against the chair.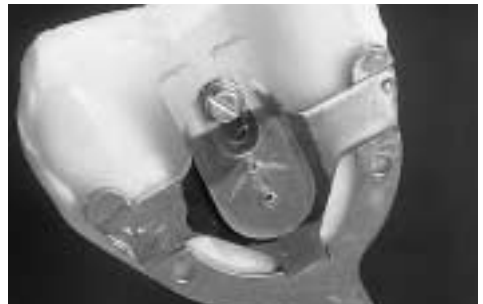

Remove the registration from the mouth and check it. If the needle-point is unclear, repeat the procedure. For fixation and final checking, mount the fixation blank in one of the two holes above the registered needle-point centric.

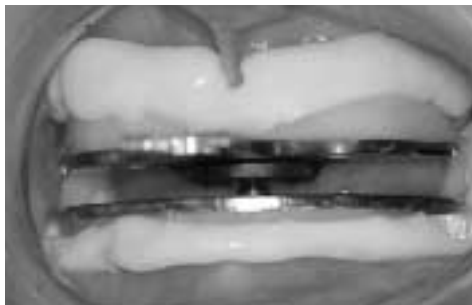Check visually in the patient's mouth to see if the registration pin meets the respective hole in the fixation blank during repeated opening and closing movements. If the pin repeatedly misses the hole, readjust the fixation blank or repeat the registration procedure. Confirm new position by repeated opening and closing movements.

Fix registration with fastsetting plaster which is filled in the space between the basic arches by means of a plastic syringe. Make sure the registration pin is located in the hole of the fixation blank. Only the lips are parted.


If no extra-oral registration is carried out with the transferbow, an impression of the lips may be taken from inside by adding plaster in the frontal area.

The individual transfer of the Bonwill triangle by means of the Universal transferbow is preferably done according to Camper's plane. The individual transfer of the Bonwill triangle by means of a transferbow helps to avoid occlusal interference, as the models may be oriented more accurately in the area of the articulator joints.


During keying the intra-oral registration, make sure the UTS adapter remains clean. If necessary, use the transferbow before keying. Insert registration joint into the UTS adapter until it snaps into place. This is the only way, the registration joint may be oriented correctly in the laboratory.

Now, mount the transferbow at patient's head according to the respective Instructions for Use. The connection with the Gnathometer M is achieved with the UTS registration joint that may newly be fixed at the Gnathometer M.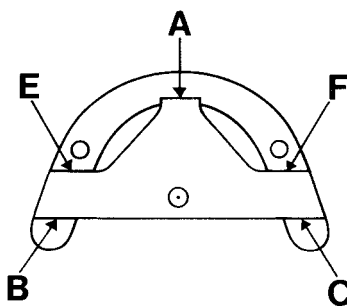


After excessive use, the clamping effect between the basic arch and registration plate may be reduced. This can be easily corrected:

The clamping effect is between point A and B/C. Points E and F always show a tolerance and do not participate in the clamping.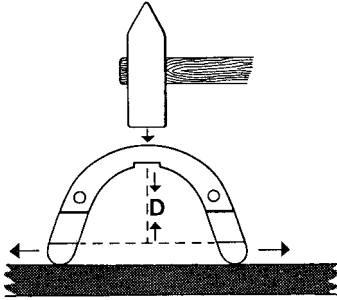

Increase the clamping effect by slightly tapping on the center of the basic arch. The distance  $D$  is reduced by fractions of a millimeter.

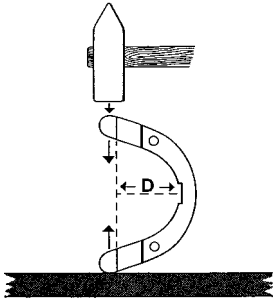If the clamping effect is too strong, reduce the distance  $D$  by fractions of a milli-meter in the illustrated way. Make sure the basic arch remains flat during this procedure.

This apparatus has been developed solely for use in dentistry. Start up and operation should be carried strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from misuse or failure to observe the Instructions. The user is solely responsible for testing the apparatus for its suitability for any purpose not explicitly stated in the Instructions.

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## **Ivoclar Vivadent UK Limited**
## **Ivoclar Vivadent, Inc.**
## **Dentigenix Inc.**