**Intended use****Intended purpose**

Protections of the tooth structure, bite adjustments, templates

**Use**

For dental use only.

**Description**

- Therapeutic appliances to correct TMJ problems and make occlusal adjustments
- Occlusal splints
- Drilling templates
- Functional try-in bodies for fixed and removable prosthetic restorations

**Indication**

Dental dysfunctions and parafunctions, partial edentulism

**Contraindications**

The material should not be used if the patient is known to be allergic to any of the constituents of ProArt Print Splint.

**Limitations of use**

- Not suitable for the fabrication of orthodontic splints
- Not suitable for use as a drill template without a guiding sleeve

**Composition**

Urethane oligomer

Triethylene glycol dimethacrylate

UDMA

BisGMA

Diphenyl(2,4,6-trimethylbenzoyl)phosphine oxide

**Application****Preparation for the printing procedure**

- ProArt Print Splint is suitable for processing in PrograPrint printers. To use the resin, remove the screw cap of the resin container and snap on the dispensing valve included in the delivery. Make sure to correctly align the valve. The valve must be in the "Closed" position. The resin container can now be inserted into the PrograPrint cartridge.
- ProArt Print Splint does not need to be mixed before the printing procedure. The silicone wiper/scrapper included in the PrograPrint printer delivery is not suitable for use in conjunction with ProArt Print Splint. For details about the insertion of the cartridge and the operation of the device, please refer to the Operating Instructions of the PrograPrint printer. To ensure the retraceability of the material batch, the batch number of each new resin container must be additionally documented.

# ProArt Print Splint

<b>SL</b> Navodila za uporabo	<b>SQ</b> Udhëzime përdorimi
- Stereolitografski material za opornice	- Material splinti me stereolitografi
<b>HR</b> Upute za uporabu	<b>RO</b> Instrucțiuni de utilizare
- Stereolitografjski materijal za udlagu	- Materiale stereolitografice pentru splint-uri
<b>CS</b> Návod k použití	<b>UK</b> Інструкція з використання
- Materiál na vytváření dlah stereolitografickým tiskem	- сплінтів методом стереолітографії
<b>SK</b> Návod na používanie	<b>ET</b> Kasutamisujuhend
- Stereolitografický dlahový materiál	- Stereolitograafia lahase materjal
<b>HU</b> Használati utasítás	<b>LV</b> Lietošanas instrukcija
- SLA (Stereolithography) sinezõ anyag	- Stereolitogrāfijas zobu aizsargu materiāls
<b>SR</b> Упутство за употребу	<b>LT</b> Naudojimo instrukcija
- Материјал за стереолитографски сплинт	- Stereolitografinė įtvaro medžiaga
<b>MK</b> Упатство за употреба	
- стереолитографски материјал за протези	
<b>BG</b> Инструкции за употреба	
- Стереолитографски материјал за шини	

# ProArt Print Splint

<b>EN</b> Instructions for Use	<b>DA</b> Brugsanvisning
- Stereolithography splint material	- Skinne og splintmateriale til stereolithografi
<b>DE</b> Gebrauchsinformation	<b>FI</b> Käyttöohjeet
- Schienen-Material für die Stereolithographie	- Stereolitografiamateriaali
<b>FR</b> Mode d'emploi	<b>NO</b> Bruksanvisning
- Matériau pour la réalisation de gouttières par stéréolithographie	- Skinnemateriale til stereolitografi
<b>IT</b> Istruzioni d'uso	<b>NL</b> Gebruiksaanwijzing
- Materiale per splintaggio per la stereolitografia	- Spalkmateriaal voor stereolithografie
<b>ES</b> Instrucciones de uso	<b>EL</b> Οδηγίες Χρήσεως
- Material para férulas realizadas mediante estereolitografía	- Υλικό για φόρτικες στερεολιθογραφίας
<b>PT</b> Instruções de Uso	<b>TR</b> Kullanma Talimatı
- Material estereolitográfico	- Stereolitografi baskı malzemesi
<b>SV</b> Bruksanvisning	<b>RU</b> Инструкция по применению
- Material till skenor framställda med stereolitograf	- материал для шин, изготовленных методом стереолитографии
<b>PL</b> Instrukcja stosowania	
- Światłoutwardzalny, jednoskładnikowy materiał łączący	

## Processing after the printing procedure

Once the printing procedure is complete, the ProArt Print Splint objects must be cleaned and post-cured. For this purpose, the objects are left to sit on the PrograPrint Stage. Cleaning and post-curing should be performed immediately after the printing process.

### Cleaning and post-curing

- The PrograPrint Clean cleaning unit is suitable for cleaning the printed objects. For detailed information about the operation of the unit, please refer to the Operating Instructions of PrograPrint Clean. When cleaning objects made from ProArt Print Splint in the PrograPrint Clean, the following parameters should be used:

	Rough cleaning	Fine cleaning
ProArt Print Splint	10 min	5 min
	850 rpm	

- After the objects have been cleaned, cautiously dry them with a gently flow of oil-free compressed air. Should sticky areas or insufficiently cleaned spots still be recognizable after air-drying, the object must be re-cleaned.
- Use the post-curing unit PrograPrint Cure to post-cure the printed objects. For detailed information about the operation of the unit, please refer to the Operating Instructions of the PrograPrint Cure.
- Once the printed objects have been successfully cleaned and post-cured, they may be removed from the build platform. For this purpose, the tools included in the PrograPrint printer delivery can be used. Detach the objects including the support structure from the PrograPrint Stage with the help of these tools.

### Finishing

- Use a side cutter to separate the supports structures from the object. Any remaining residues of the support structures should be removed using cross-cut tungsten carbide burs. These burs can also be used for shape adjustments.

### Polishing

- When polishing the objects, pay particular attention to the margins, occlusal surfaces and inner surfaces. For pre-polishing, use rubber polishers and silicone wheels with different abrasive values from coarse to fine. Polish the printed objects to a high gloss using goat hair brushes, cotton or leather buffing wheels, as well as universal polishing paste. Please note that the improper use of grinding tools, polishing brushes, steam cleaners and water baths can lead to an overheating of the object, which may result in damage to the material.
- Printed objects must be cleaned before they are inserted in the patient's mouth.

### Extensions

- Tetric EvoCeram light-curing composite, in combination with Adhese Universal, may be used to extend splints made of ProArt Print Splint in selected areas.
- Roughen the surface to be extended using diamond finishers. Then rinse the surface with the water jet and blow it dry using oil- and water-free compressed air. When applying Adhese Universal, proceed according to the instructions for the intraoral repair of fractured composite resin fillings in the Adhese Universal Instructions for Use. For guidance regarding the application of Tetric EvoCeram, please consult the Instruction for Use of the product.

## Important processing restrictions

Adhering to the following framework conditions is essential to ensure the successful processing of ProArt Print Splint:

- The instructions of the implant manufacturer must be observed when fabricating drill templates.
- The material must be processed in a compatible 3D printing system.
- For individualization only use materials which are approved or recommended.

## Safety information

- In the case of serious incidents related to the product, please contact Ivoclar Vivadent AG, Bendererstrasse 2, 9494 Schaan/Liechtenstein, website: [www.ivoclarvivadent.com](http://www.ivoclarvivadent.com), and your responsible competent authority.
- The current Instructions for Use are available in the download section of the Ivoclar Vivadent AG website ([www.ivoclarvivadent.com](http://www.ivoclarvivadent.com)).

## Warnings

- Contains (meth-)acrylates.
- Irritating to eyes, respiratory organs and skin.
- Skin contact may cause sensitization.
- Avoid skin contact with unpolymerized resin and cleaning solutions. Commercial medical gloves do not provide protection against the sensitizing effect of (meth-)acrylates.
- Do not inhale grinding dust.

## Disposal information

Liquid resin must not be disposed of with household refuse. Its disposal must take place in accordance with the respective local legal provisions. Prevent contamination of sewage systems, surface water and groundwater. Once the empty container has been thoroughly cleaned, it can be disposed of with household waste.

## Shelf life and storage

- Store the material in a cool, dark and well-ventilated place.
- Storage temperature 2–28 °C (36–82 °F)
- Do not use the product after the indicated date of expiration.
- Store opened resin containers in the closed PrograPrint cartridge.
- To avoid premature curing, liquid resin should be exposed to daylight only briefly.
- Expiration date: See note on packaging. After the container has been opened for the first time, the shelf life of the product is six months.

## Additional information

Keep material out of the reach of children!

strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of use. The user is responsible for testing the material for their suitability and use for any purpose not explicitly stated in the Instructions.

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**Ivoclar Vivadent Marketing Ltd.**
**Ivoclar Vivadent AB**
**Ivoclar Vivadent Limited**
**Ivoclar Vivadent, Inc.**