1 Intended use

Intended purpose

Cention® Forte

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Bonding of dental restorations

Patient target group

- Patients with permanent teeth
- Patients with deciduous teeth

Intended users / Special training

- Dentists
- No special training required.

Use

For dental use only.

Description

Cention® Forte

Cention® Forte is a self-curing, radiopaque filling material ($\geq 200\%$ Al) with light-curing option (Type 1, Class 3) for the direct restorative treatment of anterior and posterior teeth to restore occlusal surfaces. Cention Forte releases fluoride, calcium and hydroxide ions. It is suitable for use as a volume replacement material in the restoration of class I, II and V cavities in permanent and deciduous teeth. Cention Forte can be optionally cured with light in the wavelength range of 400 – 500 nm.

The Cention capsules are designed for capsule mixers.

The dispensable amount per mixed capsule is 0.3 g.

Cention® Primer

Cention® Primer is a self-etching, self-curing primer for tooth structure. It is specially designed for use with Cention Forte.

Indication

Missing tooth structure in anterior and posterior teeth

Contraindication

Do not use if the patient is known to be allergic to any of the ingredients of Cention Forte or Cention Primer.

Cention® Forte

Cention® Primer

EN Instructions for Use	ES Instrucciones de uso	FI Käyttöohjeet	TR Kullanma Talimatı
– Cention® Forte: Self-curing resin-based dental restorative material with light-curing option	– Cention® Forte: Material restaurativo dental de resina autopolimerizable con opción de fotopolimerización	– Cention® Forte: Kemiallisovetteinen hammastäyttemateriaali, jossa on valokovetusmahdollisuus	– Cention® Forte: Kendiliğinden sertleşen, ışıkla sertleştirme seçeneği de olan, rezin esaslı dental restoratif materyal
– Cention® Primer: Self-curing primer for Cention	– Cention® Primer: Primer autopolimerizable para Cention	– Cention® Primer: Kemiallisovetteinen Cention -primeri	– Cention® Primer: Cention için kendiliğinden sertleşen primer
DE Gebrauchsinformation	PT Instruções de Uso	NO Bruksanvisning	RU Инструкция по применению
– Cention® Forte: Selbsthärtendes zahnärztliches Füllungsmaterial mit optionaler Lichthärtung	– Cention® Forte: Material para restauração de dentária a base de resina autopolimerizável, com fotoativação opcional	– Cention® Forte: Selvhærdende odontologisk fyllingsmateriale med valgfri lysherding	– Cention® Forte: Самоотверждающий стоматологический реставрационный композиционный материал с возможностью светового отверждения
– Cention® Primer: Selbsthärtender Primer für Cention	– Cention® Primer: Primer autopolimerizável para Cention	– Cention® Primer: Selvhærdende primer for Cention	– Cention® Primer: Самоотверждающий праймер для Cention
FR Mode d'emploi	SV Bruksanvisning	NL Gebruiksaanwijzing	PL Instrukcja stosowania
– Cention® Forte: Matériau de restauration dentaire à base de résine, autopolymérisant avec option photopolymérisation	– Cention® Forte: Självhårdande resinbaserat dentalt restaurationsmaterial med ljushärdningsmöjlighet	– Cention® Forte: Zelfuithardend tandheelkundig restauratief materiaal met optie tot lichtuitharding	– Cention® Forte: Samoutwardzalny materiał do wypełnień, na bazie żywicy z opcją polimeryzacji światłem
– Cention® Primer: Primer autopolymérisant pour Cention	– Cention® Primer: Självhårdande primer för Cention	– Cention® Primer: Zelfuithardende primer voor Cention	– Cention® Primer: Samoutwardzalny primer do Cention

Limitations of use

- If a dry working field cannot be established
- If the prescribed application technique cannot be used
- Must not be used as a luting composite
- In direct pulp capping
-  For single use only. If Cention Forte / Cention Primer is applied directly in the mouth of the patient, the capsule as well as the applicator should only be used for one patient due to hygienic reasons (prevention of cross-contamination between patients).
- The product is not intended to be reprocessed or reused.

Side effects

In rare cases, ingredients of Cention Forte / Cention Primer may cause a sensitizing reaction. The product must not be used in such cases. To avoid possible irritation of the pulp, areas close to the pulp must be protected with a suitable pulp/dentin protector. Selectively apply a calcium hydroxide-based preparation in areas close to the pulp and cover it with a suitable cavity liner, if necessary.

Interactions

Peroxide-based bleaching and disinfecting agents may inhibit the polymerization of certain components of Cention Forte and Cention Primer. Materials of this kind must not be used ≤ 14 days prior to the treatment with Cention Forte & Cention Primer. Phenolic substances (e.g. eugenol) may also inhibit the polymerization of the material. Iron- or aluminium-based haemostatic agents must be rinsed away completely with plenty of water prior to the treatment with Cention Forte. Fluoride preparations must not be used prior to the treatment with Cention Forte. Discolouration may occur in combination with cationic mouthwashes, plaque disclosing agents and chlorhexidine.

Clinical benefit

- Reconstruction of chewing function
- Improvement of speech function
- Restoration of esthetics
- Pain relief

Composition

Cention Forte:

Ca-fluorosilicate glass, Ba-Al silicate glass, copolymer, Ca-Ba-Al fluorosilicate glass, UDMA, ytterbium trifluoride, aromatic aliphatic UDMA, DCP and PEG-400-DMA.

Total content of inorganic fillers: 58-59 vol%

Particle size of inorganic fillers: between 0.1 μm and 7 μm .

Cention Primer:

HEMA, MDP, Bis-GMA, D3MA, ethanol, methacrylate-modified polyacrylic acid, silicon dioxide, potassium hydroxide and campherquinone

2 Application

1. Cavity preparation

The cavity is prepared according to tooth-preserving principles. Do not prepare sharp, internal edges and angles. Do not prepare additional undercuts in caries-free areas. The geometry of the cavity is pre-determined by the dimensions of the caries lesion or the old filling. Slightly bevel or round out the enamel margins using finishing diamonds (25–40 μm). Rinse with water spray to remove all residue in the cavity. Dry the cavity with water- and oil-free air.

2. Pulp protection / Base

In very deep cavities only, areas close to the pulp should be selectively coated with a calcium hydroxide liner. Subsequently cover with a pressure resistant cement (e.g. glass ionomer cement).

3. Placement of matrix / interdental wedge

If the cavity includes proximal areas, place a matrix band and secure it with a wedge.

4. Isolation

Adequate relative or absolute isolation is required. The following auxiliary aids can be used for this purpose:

- OptraGate[®]
- OptraDam Plus

5. Usage of Cention Primer

Dispense a drop of Cention Primer into a dish and protect it from light (e.g. VivaPad[®]).

Carefully close the bottle after use. Dip the corresponding coated Cention Primer single-use applicator in the liquid and stir through for approx. 5 seconds. The mixing process is completed when the flocked applicator has a uniform yellow colour.

If this is not the case, the mixing process should be repeated. The primer must be used within 120 seconds of mixing.

A new Cention Primer single-use applicator must be used for each new application.

Note:

The activated Cention Primer single-use applicator is indicated for only one patient due to hygienic reasons (prevention of cross-contamination between patients). Any remaining activated Cention Primer must be discarded after the treatment!

Application of Cention Primer

Starting in the enamel, completely coat the tooth surfaces requiring treatment and scrub the primer in for a least 10 seconds. This interval must not be shortened. Simply spreading Cention Primer on the tooth surface is inadequate.

Disperse Cention Primer with water- and oil-free compressed air until a shiny, thin, immobile film has formed.

Note:

Any contamination of the Cention Primer single-use applicator with blood, saliva, sulcus fluid or water during the application process must be avoided. If the cavity is contaminated, it must be rinsed again and the process must be repeated.

6. Use of Cention Forte

In order to activate the capsule, press the plunger on a flat surface. Make sure that the plunger is pressed in completely to ensure thorough mixing of the powder and liquid.

The capsule must not be activated in the applicator!

Immediately after activation, insert the capsule into the capsule mixer (4000 – 4800 rpm) and mix for 15 ± 2 seconds (e.g. Silamat® S6). The mixing time is dependent on the temperature (see table).

Room/storage temperature	Mixing time
20 – 22 °C (36 – 82 °F)	17 seconds
22 – 26 °C (36 – 82 °F)	15 seconds
26 – 28 °C (36 – 82 °F)	13 seconds

After mixing, immediately remove the capsule from the device and place it into the applicator. Repeatedly activate the dispensing lever of the applicator until you can see the material in the transparent applicator tip. Then position the tip on the cavity floor and extrude the material evenly into the cavity. Keep the tip immersed in the material to prevent the formation of air bubbles. Subsequently, carefully pack and condense the material, remove occlusal excess and shape and contour the filling. The working time from the start of mixing is 2:00 minutes.

Working and setting times

Mixing time	-- 15 ± 2 seconds
Dispensing time after mixing	---- 20 seconds
Working time from the start of mixing	----- 2:00 minutes
Setting time from the start of mixing	----- 6:30 minutes

The indicated working and setting times were determined under simulated clinical conditions.

High temperatures during mixing decrease the working / setting time and dispensing time; low temperatures increase it. When using the self-curing mode, the matrix band must only be removed after the setting time has elapsed; then finishing can be started. The filling can be optionally light cured to a depth of 4 mm to speed up the curing process (exposure times according to Table). Press the release button and pull the capsule upwards to remove it from the applicator.

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ivoclar
vivadent
clinical

7. Finishing / Checking the occlusion

After polymerization, remove excess material with suitable tungsten carbide finishers or fine-grit diamonds.

Remove proximal excess with diamond or tungsten carbide finishers or using finishing strips or flexible finishing discs. Check the occlusion and articulation and grind in the necessary adjustments to prevent premature contacts or undesired articulation paths on the surface of the restoration.

Application notes

- In the case of adjustments, additional Cention Forte can be directly applied to the polymerized material. If the inhibition layer has already been removed, the filling surface must be roughened and then coated with Cention Primer before fresh Cention Forte can be applied.
- Use Cention Forte at room temperature. Cold components may be difficult to mix.

3 Safety information

- In the case of serious incidents related to the product, please contact Ivoclar Vivadent AG, Bendererstrasse 2, 9494 Schaan/Liechtenstein, website: www.ivoclarvivadent.com, and your responsible competent authority.
- The current Instructions for Use are available in the download section of the Ivoclar Vivadent AG website (www.ivoclarvivadent.com).

Warnings

Prevent any contact of uncured Cention Forte and Cention Primer with skin / mucous membrane and eyes. If Cention Primer should accidentally come in contact with the eyes, immediately rinse with plenty of water and consult a physician or eye specialist. If the product comes in contact with the skin, wash with soap and plenty of water. Unpolymerized Cention Forte and Cention Primer may have a slight irritating effect and may lead to a sensitization against methacrylates. Commercial medical gloves do not provide protection against the sensitizing effect of methacrylates.

Disposal information

- Remaining stocks must be disposed of according to the corresponding national legal requirements.

Residual risks

Users should be aware that any dental intervention in the oral cavity involves a certain risk of complications. Some of these risks are listed below:

- Failure of the adhesive bond (loss of the filling)
- Inclusion of air bubbles during the placement of the filling
- Heat development during the curing procedure
- Wear of the filling
- Chipping, fractures
- Ingestion of material
- Postoperative sensitivity

4 Shelf life and storage

- Storage temperature 2-28 °C (36–82 °F)
- If the indicated storage temperature is exceeded, proper functioning of the product cannot be ensured.
- Keep the foil bag of the Cention Forte capsule closed until use.
- Cention Primer single-use applicators must be stored in the plastic container provided. Keep the plastic container closed.
- Do not use the product after the indicated date of expiration.
- Expiry date: See note on the packages.

5 Additional information

Keep material out of the reach of children!
Not all products are available in all countries.

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of use. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions.

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Ivoclar Vivadent Marketing Ltd.
Ivoclar Vivadent AB
Ivoclar Vivadent Limited
Ivoclar Vivadent, Inc.