# Syntac®

# Syntac®

English

Brugsanvisning

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Bruksanvisning

Productinformatie

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For dental use only.  
Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed dentist.

Made in Liechtenstein  
Ivoclar Vivadent AG,  
FL-9494 Schaan / Liechtenstein

CE 0123

ivoclar  
vivadent®

## Instructions for Use

### Description

Syntac is a proven enamel-dentin adhesive system for direct and indirect restorations. It can be used in combination with both light- and self-curing composites. The light-curing bonding agent Heliobond must be used prior to the application of the composite.

### Composition

*Syntac Primer* contains triethylene glycol dimethacrylate, polyethylene glycol dimethacrylate, maleic acid and acetone in an aqueous solution.

*Syntac Adhesive* contains polyethylene glycol dimethacrylate and glutaraldehyde in an aqueous solution. *Heliobond* contains Bis-GMA, triethylene glycol dimethacrylate, stabilizers and catalysts.

### Indications

- Adhesive for direct composite restorations
- Adhesive for indirect restorations

### Contraindications

Do not use Syntac or Heliobond if a patient is known to be allergic to any of the materials' ingredients or if the stipulated working technique cannot be employed.

### Side effects

In rare cases, components of Syntac or Heliobond may lead to sensitization. In these cases, Syntac or Heliobond should no longer be used.

### Interactions

Materials containing eugenol or clove oil may inhibit the polymerization of Heliobond.

602960410/WE3H

## Application

### Note:

**Do not mix Syntac Primer and Syntac Adhesive.**

**Apply the products sequentially!**

1. A dry operating field must be ensured. Ideally, a rubber dam or the user-friendly OpraDam® Plus should be used.
2. In very deep cavities, areas close to the pulp should be selectively coated with a calcium hydroxide liner (e.g. ApexCal) and subsequently covered with pressure-resistant cement (glass ionomer cement, e.g. Vivaglass Liner).
3. Apply phosphoric acid gel (e.g. Total Etch) first to the prepared enamel and then to the dentin surfaces. The etchant should be left to react on the enamel for 15 to 30 seconds, and on the dentin for 10 to 15 seconds. Dentin etching is an optional working step and is not required for Syntac. Following this, remove all etchant gel with a vigorous water spray for at least 5 seconds. Excess moisture should be removed leaving the dentin surface with a slightly glossy wet appearance (wet bonding).  
Excess moisture should be removed with either air syringe, a dry brush, a foam pellet or other lint-free absorbents.

### Do not overdry the dentin!

4. Apply Syntac Primer with the brush on the cavity and gently rub in. Syntac Primer requires a contact time of at least 15 seconds on the dentin.
5. Disperse excess Syntac Primer and dry thoroughly. Do not rinse.
6. Apply Syntac Adhesive, leave it for 10 seconds and thoroughly dry the cavity with an air syringe. Do not rinse.
7. Apply Heliobond and blow to a thin layer.

### Note:

At this point, Heliobond must be light-cured for direct restorations. For the placement of indirect restorations, Heliobond is polymerized together with the cementation material.

8. Light-cure Heliobond for at least 10 seconds with a polymerization light using a light intensity of at least 500 mW/cm<sup>2</sup> (e.g. bluephase in the LOP mode).
9. Apply the restorative (e.g. Tetric EvoCeram) or cementation material (e.g. Variolink II) according to the instructions of the manufacturer.

## Special notes

- A Syntac does not contain any polymerization initiators. Therefore, the subsequent application of the light-curing bonding agent Heliobond is mandatory.
- B When cementing lab-fabricated inlays, crowns or veneers with low-viscosity, light- or dual-curing luting composites (e.g. Dual Cement or Variolink II), Heliobond should not be light-cured separately. Heliobond should be polymerized together with the luting composite (Dual Cement or Variolink II) to assure the accurate fit of the restoration.
- C If self-curing composites are used, the light-curing bonding agent Heliobond must be applied. The bonding agent must be light-cured before the self-curing composite is applied.
- D The solvent contained in Syntac Primer (acetone) is highly volatile. If this solvent evaporates, the monomer separates at 23 °C/73 °F and a mean drop size after about 60 seconds. Therefore, the primer should be dispensed from the bottle immediately before application. If several cavities are being restored, fresh primer should be dispensed for each cavity. Do not use the primer after the mentioned separation process has taken place.

## Warnings

- Syntac Primer is an irritant and highly flammable.
- Heliobond is an irritant.
- Avoid contact of Syntac Primer, Syntac Adhesive or Heliobond with skin, mucous membrane and eyes. If the product comes in contact with the skin, rinse with soap and water.
- If Syntac comes in contact with the eyes, immediately rinse with copious amounts of water for at least 15 minutes and consult a physician/ophthalmologist.
- Syntac Adhesive contains glutaraldehyde and is detrimental to health if inhaled or swallowed.
- In rare cases, contact with skin may lead to a sensitization to methacrylates and glutaraldehyde.
- Commercial medical gloves do not provide protection against the sensitizing effect of methacrylates.

## Storage

- Expiry date: see note on bottle/packaging.
- Immediately close the bottle with the original cap after use.
- Storage temperature: 2–28 °C (36–82 °F).
- Do not use the material if Syntac Primer shows clear signs of a change in consistency.

## For use in dentistry only!

### Keep out of the reach of children!

## Date information prepared:

04/2010; Rev. 2

## Manufacturer:

Ivoclar Vivadent AG  
FL-9494 Schaan / Liechtenstein

This material has been developed solely for use in dentistry. Processing should be done strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions.

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## **Ivoclar Vivadent UK Limited**
## **Ivoclar Vivadent, Inc.**