Instructions for Use

Mode d'emploi

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For dental use only.

Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed dentist.

Made in Liechtenstein

Ivoclar Vivadent AG, FL-9494 Schaan/Liechtenstein

CE 0123

ivoclar
vivadent®
clinical

633054/1 2/09/16/spr/G

Instructions for Use

Description

Metal/Zirconia Primer is a single-component primer promoting an adhesive bond between luting composites (particularly Multilink N) and indirect restorations made of zirconium oxide and aluminium oxide ceramic or metal and metal-ceramic.

Composition

Phosphonic acid methacrylate and methacrylate crosslinking agent in an organic solvent.

Indication

Metal/Zirconia Primer serves as bonding agent and is used to create a durable chemical bond between luting composites and zirconium oxide, aluminium oxide, metal and metal-ceramic restorations.

Contraindications

- Any use not listed in the indications.
- If a patient is known to be allergic to any of the ingredients of Metal/Zirconia Primer.

Side effects

To date, no systemic side effects are known.

In rare cases, components of Metal/Zirconia Primer may lead to sensitization. Metal/Zirconia Primer should not be used in such cases.

Interactions

Contamination with saliva, blood or water must absolutely be avoided during the entire procedure.

Application

- 1 Adhesive cementation of prosthetic restorations:
 - 1.1 Preparing the restoration
 - 1.1.1 Zirconium oxide (e.g. IPS e.max ZirCAD) or aluminium oxide restorations

- Sandblast the restoration surface (sandblasting parameters according to the instructions of the manufacturer of the restorative material).
 - If necessary, clean the restoration in an ultrasonic unit for about 1 minute.
 - Thoroughly rinse with water spray and dry with oil-free air.
 - **IMPORTANT!** In order to create a strong bond, do not clean the oxide ceramic surfaces with phosphoric acid.
- 1.1.2 Metal or metal-supported restorations
- Sandblast the restoration surface (sandblasting parameters according to the instructions of the manufacturer of the restorative material) until an even, matt or non-glossy surface has been achieved.
 - If necessary, clean the restoration in an ultrasonic unit for about 1 minute.
 - Thoroughly rinse with water spray and dry with oil-free air.
 - **IMPORTANT!** In order to create a strong bond, do not clean the metal surfaces with phosphoric acid.
- 1.2 Apply Metal/Zirconia Primer with a brush or a micro-brush to the pre-treated surfaces. Allow the material to react for 180 seconds. Disperse with a strong stream of air.
- 1.3 Apply the luting composite (e.g. Multilink N) according to the respective Instructions for Use and insert the restoration.

Special notes

- Contamination with saliva, blood or water must absolutely be avoided during the entire procedure. In case of contamination, the entire treatment of the restoration is to be repeated (see section 1).
- If the primer dries out on the surface of the restoration before blasting with a strong stream of air, the surface needs to be rinsed with pure alcohol and blown with air.
- Do not insert the brush into the Metal/Zirconia Primer bottle. This may lead to contamination.
- Carefully close the Metal/Zirconia Primer bottle immediately after use.

Warning

- Flammable
- The product contains flammable organic solvents. Keep away from ignition sources.
- Metal/Zirconia Primer is an irritant. Avoid contact with the skin and mucous membrane. If the material should accidentally come in contact with the skin, immediately wash with copious amounts of water.
- In rare cases, contact with the skin may lead to sensitization towards the ingredients.
- Commercial medical gloves do not provide protection against the sensitizing effect of methacrylates.

Storage

- Metal/Zirconia Primer should never be used after the expiration date.
- Storage temperature: 2–8 °C / 36–46 °F.
- Shelf life: see expiry date on label.

**Keep out of the reach of children.
For use in dentistry only.**

Date information prepared: 12/2009, Rev. 0

Manufacturer

Ivoclar Vivadent AG
FL-9494 Schaan / Liechtenstein

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions for Use or the stipulated area of application. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions for Use.

**The Material Safety Data Sheet is available online at
www.ivoclarvivadent.com**

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Ivoclar Vivadent Limited
Ivoclar Vivadent, Inc.