Description

Cervitec® Plus is a varnish that protects sensitive dentin in the case of exposed cervicals. Chlorhexidine and thymol are the antimicrobial components in Cervitec Plus. They protect the tooth surface by reducing bacterial activity.

Intended purpose

Protection of teeth

Generally, Cervitec Plus is applied every three months. If intensive treatment is required, however, the varnish may also be applied more frequently.

Composition

Ethanol, water, acrylate copolymer, vinyl acetate copolymer and

Chlorhexidine diacetate 1 %

Thymol 1 %

Indication

- Exposed root surfaces
- Hypersensitive cervicals
- Bacterial activity on tooth surfaces

Contraindication

If patients are known to be allergic to any of the ingredients in Cervitec Plus, the material should not be applied.

Warning

In rare, isolated cases, chlorhexidine may cause allergic reactions.

Side effects

In individual cases, contact with the mucous membrane may result in short-term, reversible irritation.

Chlorhexidine may cause reversible discolouration of the tooth structure and restorations.



Application

Cervitec Plus can be used for patients of all age groups and is professionally applied by dentists, dental hygienists or dental prophylaxis assistants.

One Free Stand Single Dose Cervitec Plus is enough for the treatment of one set of teeth. One tube is enough for about 20 to 30 sets of teeth.

Step-by-step application procedure

1. Clean tooth surfaces thoroughly.
2. Dry with cotton rolls and air syringe.
3. a) **Single Dose:**
Remove the foil from the Single Dose and apply the varnish directly from the Single Dose.
- b) **Tube:**
Press out 3 drops into, for example, a dappen dish and reseal the tube. Hold the tube in a vertical position with the cap facing upwards for opening and closing. For pressing out the contents, hold the opening vertically down.
4. Apply a thin coat of varnish by means of a Vivadent applicator or suitable brush (diagram); in proximal areas, Cervitec Plus is applied with dental floss.
5. Let the varnish dry. The varnish can be dispersed with air.
6. Remove cotton rolls after 30 seconds.
7. Ask patients not to rinse their mouth.

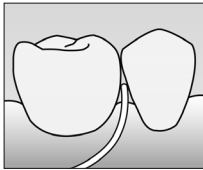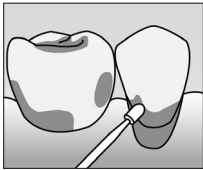**Tips for patients**

After the application of Cervitec Plus, patients should not eat or drink for 1 hour.

Special note

Operators can eliminate the risk of cross-contamination by using Cervitec Plus in the Single-Dose vessel for just one patient. The applicator is for single use only and cannot be sterilized.

Storage

- Close the tube immediately after use
- Storage at temperature 2-8 °C / 36-46 °F
- Highly flammable. Keep away from sources of ignition
- Expiry date: see label / packaging

Keep material out of children's reach.

For use in dentistry only.

For U.S.: Rx only

This material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damage resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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Ivoclar Vivadent Marketing Ltd.
Ivoclar Vivadent AB
Ivoclar Vivadent Limited
Ivoclar Vivadent, Inc.