### Instructions for Use Gebrauchsinformation

Mode d'emploi

Istruzioni d'uso

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Bruksanvisning

Brugsanvisning

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Bruksanvisning

Productinformatie

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For dental use only!

Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed dentist.

Made in Liechtenstein

Ivoclar Vivadent AG, FL-9494 Schaan/Liechtenstein

CE 0123

564348/0510/WE3/H

ivoclar  
vivadent®  
clinical

## Instructions for Use

### Description

VivaSens is a liquid desensitizer for the treatment of hypersensitive root surfaces.

### Composition

Varnish (ethanol, water and hydroxypropylcellulose) containing potassium fluoride, polyethyleneglycol dimethacrylate, and other methacrylates.

### Indication

- Treatment of hypersensitive cervicals
- Treatment of tooth sensitivities e.g. due to bleaching procedures
- Treatment of sensitivities occurring after professional tooth cleaning, scaling and root- planing procedures
- Dentin-protection below restorative materials

### Contraindication

- If a patient is known to be allergic or hypersensitive to any of the material's ingredients
- If the sensitivity arises from pulpitis or other dental disease
- Pulp-capping with VivaSens

### Side Effects

In individual cases contact with the mucous membranes may result in short-term reversible irritation. Certain ingredients of VivaSens may cause sensitization in predisposed patients or dental personnel which can lead to allergic contact dermatitis. In such cases, VivaSens should not be used.

### Interactions

If VivaSens is used to protect dentin before adhesive restorative treatment, compatibility with the chosen adhesive must be considered. If used with the Total Etch adhesive ExcITE®, VivaSens may be applied after the etching step and always prior to the adhesive. Interactions with other Total Etch adhesives (using the same treatment procedure) are not to be expected but cannot be excluded. Compatibility with other types of adhesives must be checked by the user.

## Application

VivaSens Free Stand® Single Dose is applied by professional dental personnel, using the pre-mounted and coated brush provided. Do not use one single dose for more than one patient!

### Step by step application procedure

#### 1.a Free Stand® Single Dose

- Remove the foil lid.
- Remove the white applicator brush from its chamber and dip into the chamber containing the VivaSens liquid.
- Twirl the applicator around thoroughly in order to mix the liquid with the ingredients contained on the applicator brush.

#### 1.b Bottle

- Squeeze 3 drops of VivaSens onto a mixing block.
- Mix the liquid well with one of the pre-coated applicator brushes (Always use exactly 3 drops per brush)

2. Clean all tooth surfaces thoroughly.
3. Dry the tooth surfaces with cotton pellets or a gentle stream of air.
4. Apply VivaSens to the areas of the teeth affected by hypersensitivity, using the disposable brush provided. Gently rub the liquid into the teeth for at least 10 seconds, avoiding contact with the gingiva.
5. Evenly disperse the liquid and dry by gently blowing air on the treated surfaces for 10 seconds.

## Patient instruction

Advise patients not to rinse, eat, drink or brush their teeth for at least 30 minutes following treatment.

## Tip

Repeat the application if sensitivity recurs or persists.

## Warning

- VivaSens liquid is an irritant. The coating on the applicator brush is corrosive. Contact with the skin, mucous membranes and eyes should be avoided.
- If contact with the eyes should occur, rinse immediately with water and consult a doctor/ophthalmologist.  
If the product comes into contact with the skin, wash thoroughly with soap and water.

- Repeated skin contact may cause sensitivity to methacrylates.
- Standard medical gloves do not offer protection from sensitization to methacrylates.
- Easily flammable. VivaSens contains ethanol and must be kept away from sources of ignition.

## Special note

Operators can eliminate the risk of cross-contamination by using VivaSens in the Single-Dose vessel for just one patient. The applicator is for single use only and cannot be sterilized.

## Storage

- Shelf-life: see expiry date on the Single Dose
- Do not use material after the indicated date of expiration
- Storage temperature: 2–28°C (36–82°F)

**Keep the material out of the reach of children!**

**For dental use only!**

**Date information prepared:** 04/2010, Rev. 1

## Manufacturer

Ivoclar Vivadent AG, FL-9494 Schaan/Liechtenstein

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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## **Ivoclar Vivadent UK Limited**
## **Ivoclar Vivadent, Inc.**
## **Dentigenix Inc.**