Light Tray

Instructions for Use
Gebrauchsinformation
Mode d'emploi
Istruzioni d'uso
Instrucciones de uso
Instruções de Uso

For dental use only!

Rx only

Manufacturer:

Ivoclar Vivadent AG, 9494 Schaan/Liechtenstein
www.ivoclarvivadent.com

Date information prepared: 2014-10/Rev. 1

WE1/2014-11-10

ivoclar[®]
vivadent[®]
technical

Ivoclar Vivadent – worldwide

Ivoclar Vivadent AG
Bendererstrasse 2
9494 Schaan
Liechtenstein
Tel. +423 235 35 35
Fax +423 235 33 60
www.ivoclarvivadent.com

Ivoclar Vivadent Pty. Ltd.
1 – 5 Overseas Drive
P.O. Box 367
Noble Park, Vic. 3174
Australia
Tel. +61 3 9795 9599
Fax +61 3 9795 9645
www.ivoclarvivadent.com.au

Ivoclar Vivadent GmbH
Tech Gate Vienna
Donau-City-Strasse 1
1220 Wien, Austria
Tel. +43 1 263 191 10
Fax: +43 1 263 191 111
www.ivoclarvivadent.at

Ivoclar Vivadent Ltda.
Alameda Caiapós, 723
Centro Empresarial Tamboré
CEP 06460-110 Barueri – SP
Brazil
Tel. +55 11 2424 7400
Fax +55 11 3466 0840
www.ivoclarvivadent.com.br

Ivoclar Vivadent Inc.
1-6600 Dixie Road
Mississauga, Ontario
L5T 2Y2, Canada
Tel. +1 905 670 8499
Fax +1 905 670 3102
www.ivoclarvivadent.us

Ivoclar Vivadent Shanghai Trading Co., Ltd.
2/F Building 1, 881 Wuding Road, Jing An District
200040 Shanghai
China
Tel. +86 21 6032 1657
Fax +86 21 6176 0968
www.ivoclarvivadent.com

Ivoclar Vivadent Marketing Ltd.
Calle 134 No. 7-B-83, Of. 520
Bogotá
Colombia
Tel. +57 1 627 3399
Fax +57 1 633 1663
www.ivoclarvivadent.co

Ivoclar Vivadent SAS
B.P. 118
F-74410 Saint-Jorioz
France
Tel. +33 4 50 88 64 00
Fax +33 4 50 68 91 52
www.ivoclarvivadent.fr

Ivoclar Vivadent GmbH
Dr. Adolf-Schneider-Str. 2
D-73479 Ellwangen, Jagst
Germany
Tel. +49 7961 889 0
Fax +49 7961 6326
www.ivoclarvivadent.de

Wieland Dental + Technik GmbH & Co. KG
Lindenstrasse 2
75175 Pforzheim
Germany
Tel. +49 7231 3705 0
Fax +49 7231 3579 59
www.wieland-dental.com

Ivoclar Vivadent Marketing (India) Pvt. Ltd.
503/504 Raheja Plaza
15 B Shah Industrial Estate
Veera Desai Road, Andheri
(West) Mumbai, 400 053
India
Tel. +91 22 2673 0302
Fax +91 22 2673 0301
www.ivoclarvivadent.in

Ivoclar Vivadent s.r.l.
Via Isonzo 67/69
40033 Casalecchio di Reno
(BO), Italy
Tel. +39 051 6113555
Fax +39 051 6113565
www.ivoclarvivadent.it

Ivoclar Vivadent K.K.
1-28-24-4F Hongo
Bunkyo-ku
Tokyo 113-0033
Japan
Tel. +81 3 6903 3535
Fax +81 3 5844 3657
www.ivoclarvivadent.jp

Ivoclar Vivadent Ltd.
12F W-Tower, 1303-37
Seocho-dong, Seocho-gu,
Seoul 137-855
Republic of Korea
Tel. +82 2 596 0714
Fax +82 2 596 0155
www.ivoclarvivadent.co.kr

Ivoclar Vivadent S.A. de C.V.
Av. Insurgentes Sur No. 863,
Piso 14, Col. Napoles
03810 México, D.F.
México
Tel. +52 55 5062 1000
Fax +52 55 5062 1029
www.ivoclarvivadent.com.mx

Ivoclar Vivadent BV
De Fruittuinen 32
2132 NZ Hoofddorp
Netherlands
Tel. +31 23 529 3791
Fax +31 23 555 4504
www.ivoclarvivadent.com

Ivoclar Vivadent Ltd.
12 Omega St, Rosedale
PO Box 303011
North Harbour
Auckland 0751
New Zealand
Tel. +64 9 914 9999
Fax +64 9 914 9990
www.ivoclarvivadent.co.nz

Ivoclar Vivadent Polska Sp. z o.o.
Al. Jana Pawla II 78
00-175 Warszawa, Poland
Tel. +48 22 635 5496
Fax +48 22 635 5469
www.ivoclarvivadent.pl

Ivoclar Vivadent Marketing Ltd.
Prospekt Andropova 18 korp. 6/
office 10-06
115432 Moscow
Russia
Tel. +7 499 418 0300
Fax +7 499 418 0310
www.ivoclarvivadent.ru

Ivoclar Vivadent Marketing Ltd.
Qlaya Main St. Siricon Building
No.14, 2nd Floor
Office No. 204
P.O. Box 300146
Riyadh 11372
Saudi Arabia
Tel. +966 11 293 8345
Fax +966 11 293 8344
www.ivoclarvivadent.com

Ivoclar Vivadent S.L.U.
Carretera de Fuencarral nº24
Portal 1 – Planta Baja
28108-Alcobendas (Madrid)
Spain
Telf. +34 91 375 78 20
Fax: +34 91 375 78 38
www.ivoclarvivadent.es

Ivoclar Vivadent AB
Dalvägen 14
S-169 56 Solna
Sweden
Tel. +46 8 514 939 30
Fax +46 8 514 939 40
www.ivoclarvivadent.se

Ivoclar Vivadent Liaison Office
: Tesvikiye Mahallesi
Sakayik Sokak
Nisantas' Plaza No:38/2
Kat:5 Daire:24
34021 Sisli – Istanbul
Turkey
Tel. +90 212 343 0802
Fax +90 212 343 0842
www.ivoclarvivadent.com

Ivoclar Vivadent Limited
Ground Floor Compass Building
Feldspar Close
Warrens Business Park
Enderby
Leicester LE19 4SE
United Kingdom
Tel. +44 116 284 7880
Fax +44 116 284 7881
www.ivoclarvivadent.co.uk

Ivoclar Vivadent, Inc.
175 Pineview Drive
Amherst, N.Y. 14228
USA
Tel. +1 800 533 6825
Fax +1 716 691 2285
www.ivoclarvivadent.us

ivoclar[®]
vivadent[®]
technical

Description

Light Tray is a ready-to-use, light-curing resin.

The light-proof container permits optimum utilization of the material. Surplus material can be stored away from light ready for reuse.

Composition

Light Tray consists of acrylated oligomers, glass fillers, dimethacrylate, silicon dioxide, catalysts and pigments.

Indications

Light Tray can be used for the fabrication of:

- customized trays
- bite plates
- base plates for wax try-ins
- base plates for fixing registration devices
- check bites for framework try-ins

Contraindications

Avoid direct intraoral contact with unpolymerized Light Tray material.

Side effects

In patients with a hypersensitivity, repeated contact with unpolymerized material may cause irritation or, in rare cases, lead to a sensitizing reaction. It should not be used for patients with a known allergy to any of the material's ingredients.

Features

- quick, clean application
- exact fit
- excellent adaptability
- high stability and good elastic recovery
- uniform layer thickness
- economical, thanks to reuse of surplus material

Instructions for Use

After blocking out undercut areas, soak the model in water and isolate with Separating Fluid (Ivoclar Vivadent). Mould Light Tray onto the model using a suitable instrument.

Working time

In normal daylight without direct exposure to sunlight: approx. 10–15 minutes (take the lighting in your working area into consideration)

Wet the instrument with a little water to prevent the material from adhering to it. Cut the functional edge to length with a sharp instrument and remove excess. Use the surplus material to build the tray grip. Residual Light Tray material can be reused at a later date. Please keep in mind that exposure to ultraviolet light for an extended period of time can trigger polymerization!

Polymerize the completed tray in a suitable polymerization unit. Any commercially available ultraviolet or halogen light-polymerization unit can be used. Since the light output intensity of such units varies considerably, the material should be cured from both sides without vacuum for 3–5 minutes. Finishing can be done immediately after curing.

Optimum physical properties are achieved after a **polymerization time of 5 minutes**. Use alcohol to clean the polymerized surface. If necessary, the functional edge can be trimmed with an abrasive instrument. Because Light Tray produces a very fine dust when ground it is advisable to work with a suction device and a mask. Use a pumice powder slurry for polishing.

The tray mad with Light Tray can be used immediately for impression taking.

Modifications and corrections

Remove surplus, unpolymerized Light Tray material from the light-protected container. Roughen or sandblast the surface to be supplemented and clean it. Adapt the material well and polymerize it again.

Impression materials

All impression materials can be used with the appropriate adhesive – provided that they do not require moist storage.

Suitable polymerization units

- Lumamat 100 (Ivoclar Vivadent) – 10 min. Program 2
- Any other commercially available ultraviolet or halogen light-polymerization unit.

Important

- **Place excess material back into the light-protective packaging immediately!**
- Material demonstrating signs of polymerization should be discarded!
- Remove the soft inhibited surface layer with alcohol!

Storage

- Store protected from light.
- Storage temperature: 2–28 °C / 36–82 °F.
- Do not use the material after the indicated date of expiration.
- Keep out of children's reach.

Warning

Unpolymerized Light Tray material may have an irritating effect. Avoid contact with skin mucous membrane and eyes. Direct skin contact with unpolymerized material may lead to a sensitizing reaction. A mask should be worn and a suction device used during grinding.

This material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions.

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