Description

VivaStyle Paint On Plus is a brush-on whitening varnish for lightening vital and non-vital teeth. The product is administered by the user.

Composition

Alcohol, Ethylcellulose, 6% Hydrogen Peroxide, Aqua, Panthenol, Aroma.
Active ingredient: 6% hydrogen peroxide.

One stand-tube contains 17 g varnish. This corresponds to 20 ml.

Indication

- Whitening of discoloured vital teeth
- Whitening of discoloured non-vital teeth

Contraindication

VivaStyle Paint On Plus should not be used

- during pregnancy or lactation;
- by children;
- by heavy smokers;
- by people who consume large quantities of alcohol;
- by people who are known to be allergic to any of the material's ingredients;
- in the presence of cervically exposed teeth;
- in the presence of untreated primary or secondary caries;
- in the presence of restorations with marginal gaps.

Applicable to EU countries:

Contains hydrogen peroxide. Avoid contact with eyes. Rinse immediately if product comes into contact with them. Not to be used on a person under 18 years of age. To be only sold to dental practitioners. For each cycle of use, the first use to be only done by dental practitioners or under their direct supervision if an equivalent level of safety is ensured. Afterwards to be provided to the consumer to complete the cycle of use. Keep out of the reach of children.

Side effects

Temporary irritation of the mucous membrane or sensitivity of the teeth may occur. These side effects usually subside within a short time after the treatment is finished or discontinued. Ingestion may result in temporary irritation of the stomach.

Application

- Before VivaStyle Paint On Plus is used, the dental professional should carefully examine the patient's medical history, inform the patient about the application procedure and clean the patient's teeth professionally. VivaStyle Paint On Plus is administered by the patient at home as instructed by the dental profession. It is advisable for the dental professional to monitor the whitening process.
- The first application of any tooth whitening cycle should be performed by a dentist or under their supervision, provided the same level of safety is guaranteed. Subsequently, the varnish must be placed at the disposal of the patient to complete the bleaching cycle at home.
- Individual teeth as well as an entire row of teeth can be whitened.

VivaStyle® Paint On Plus

VivaStyle® Paint On Plus

EN	Instructions for Use Tooth whitening varnish	FI	Käyttöohjeet Hamppaiden valkaisulakka	RU	Инструкция по применению Лак для отбеливания зубов
DE	Gebrauchsinformation Lack zur Zahnaufhellung	NO	Bruksanvisning Lakk til bleking av tenner	UK	Інструкції щодо використання Лак для відбілювання зубів
FR	Mode d'emploi Vernis de blanchiment dentaire	NL	Gebruiksaanwijzing Lak voor het witter maken van tanden	ET	Kasutusjuhised Hammasete valgenduslakk
IT	Istruzioni d'uso Lacca per lo sbiancamento	EL	Οδηγίες Χρήσεως Βερνίκι λεύκανσης δοντιών	LV	Lietošanas instrukcija Zobu balināšanas laka
ES	Instrucciones de uso Barniz de blanqueamiento dental	TR	Kullanma Talimatı Diş beyazlatma jeli	LT	Naudojimo instrukcijos Dantų balinimo lakas
PT	Instruções de Uso Verniz para clareamento dental	SL	Navodila za uporabo Lak za beljenje zob		
SV	Bruksanvisning Lack för tandblekning	HR	Upute za uporabu Lak za izbjeljivanje zubi	BG	Инструкции за употреба Лак за избелване на зъби
DA	Bruksanvisning Lak til tandblegning	CS	Návod k použití Lak na bělení zubů	SQ	Udhëzimet e përdorimit Lustër për zbardhjen e dhëmbëve
				RO	Instrucțiunile de utilizare Lac de albire a dinților

Step-by-step instructions

1. Clean teeth thoroughly. Use dental floss to clean between the teeth. Rinse well.
2. Fill the dispensing cup up to the brim with VivaStyle Paint On Plus. The dispensed amount is usually enough for treating the teeth of both the upper and lower jaw.
3. Pull the lips up and to the side so that the teeth are clearly visible.
4. Use a soft absorbent paper towel to blot dry the facial surfaces of the upper teeth. Keep retracting the lips so that the tooth surface remains dry.
5. Apply a thin even layer of VivaStyle Paint On Plus to the front teeth with the brush. Paint along the sides of the teeth as well as along the gumline. Make sure that the varnish does not get on the gums. Retract the lips for another 30 seconds to enable the varnish to dry. The varnish has a matt white appearance when it has set.
6. Subsequently, dry the lower teeth and repeat the above steps up to the stage where the varnish is left to dry.
7. Close the mouth. The varnish is left on the teeth for 10 minutes. Note: Do not eat, drink or smoke during this time.
8. After 10 minutes, remove the varnish by brushing the teeth with a soft toothbrush without tooth paste. Rinse thoroughly. Spit everything out – do not swallow.
9. Use VivaStyle Paint On Plus daily without skipping a day. The varnish may be applied once or twice a day, depending on how the treatment fits into the daily schedule. A new dispensing cup and brush should be used for every application.

Notes

The following points should be noted in particular during the treatment:

- Avoid consuming citrus fruit and fruit juices, as they can promote hypersensitivity.
- Avoid, or at least minimize, the consumption of coffee, tea, red wine and colas as well as smoking to prevent the risk of staining the teeth.
- Only natural teeth are whitened. The appearance of existing restorations in the front teeth may no longer match that of the whitened teeth. Therefore, these restorations may have to be replaced.
- The treatment with VivaStyle Paint On Plus should be completed at least two weeks prior to the placement of tooth-coloured restorations.
- The teeth cannot be made to look whiter than their natural colour.
- The application of a thicker layer or several layers of varnish does not improve the results.
- The set varnish may appear to coat the teeth unevenly.

If you have any questions about the treatment or if you experience uncomfortable sensitivity, please contact your dental office.

Warning

Avoid contact of the material with the skin, mucous membrane and eyes. After accidental contact, rinse the affected area with copious amounts of water. If contact with the eyes occurs, immediately rinse with plenty of water and seek medical advice.

Storage

- VivaStyle Paint On Plus is highly flammable
- Protect VivaStyle Paint On Plus from heat and sunlight
- Do not use after the date of expiration
- Store at 2 – 28 °C / 36 – 82 °F.

Keep out of the reach of children!

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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Alcohol, Ethylcellulose, 6% Hydrogen Peroxide, Aqua, Panthenol, Aroma.

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www.ivoclarvivadent.com

Ivoclar Vivadent Pty. Ltd.

1 – 5 Overseas Drive, P.O. Box 367, Noble Park, Vic. 3174, Australia

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www.ivoclarvivadent.us

Ivoclar Vivadent Shanghai Trading Co., Ltd.

2/F Building 1, 881 Wuding Road, Jing An District, 200040 Shanghai, China

Tel. +86 21 6032 1657, Fax +86 21 6176 0968

www.ivoclarvivadent.com

Ivoclar Vivadent Marketing Ltd.

Calle 134 No. 7-B-83, Of. 520, Bogotá, Colombia

Tel. +57 1 627 3399, Fax +57 1 633 1663

www.ivoclarvivadent.co

Ivoclar Vivadent SAS

B.P. 118, 74410 Saint-Jorioz, France

Tel. +33 4 50 88 64 00, Fax +33 4 50 68 91 52

www.ivoclarvivadent.fr

Ivoclar Vivadent GmbH

Dr. Adolf-Schneider-Str. 2, 73479 Ellwangen, Jagst, Germany

Tel. +49 7961 889 0, Fax +49 7961 6326

www.ivoclarvivadent.de

Ivoclar Vivadent Marketing (India) Pvt. Ltd.

503/504 Raheja Plaza, 15 B Shah Industrial Estate

Veera Desai Road, Andheri (West), Mumbai, 400 053, India

Tel. +91 22 2673 0302, Fax +91 22 2673 0301

www.ivoclarvivadent.in

Ivoclar Vivadent Marketing Ltd.

The Icon, Horizon Broadway BSD, Block M5 No. 1

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www.ivoclarvivadent.com

Ivoclar Vivadent Ltd.

12 Omega St, Rosedale, PO Box 303011 North Harbour

Auckland 0751, New Zealand

Tel. +64 9 914 9999, Fax +64 9 914 9990

www.ivoclarvivadent.co.nz

Ivoclar Vivadent Polska Sp. z o.o.

Al. Jana Pawła II 78, 00-175 Warszawa, Poland

Tel. +48 22 635 5496, Fax +48 22 635 5469

www.ivoclarvivadent.pl

Ivoclar Vivadent LLC

Prospekt Andropova 18 korp. 6/, office 10-06, 115432 Moscow, Russia

Tel. +7 499 418 0300, Fax +7 499 418 0310

www.ivoclarvivadent.ru

Ivoclar Vivadent Marketing Ltd.

Qlaya Main St., Siricon Building No.14, 2nd Floor, Office No. 204

P.O. Box 300146, Riyadh 11372, Saudi Arabia

Tel. +966 11 293 8345, Fax +966 11 293 8344

www.ivoclarvivadent.com

Ivoclar Vivadent S.L.U.

Carretera de Fuencarral nº24, Portal 1 – Planta Baja
28108-Alcobendas (Madrid), Spain
Tel. +34 91 375 78 20, Fax +34 91 375 78 38

www.ivoclarvivadent.es

Ivoclar Vivadent AB

Dalvägen 14, 169 56 Solna, Sweden
Tel. +46 8 514 939 30, Fax +46 8 514 939 40

www.ivoclarvivadent.se

Ivoclar Vivadent Liaison Office

: Tesvikiye Mahallesi, Sakayik Sokak, Nisantas' Plaza No:38/2
Kat:5 Daire:24, 34021 Sisli – Istanbul, Turkey
Tel. +90 212 343 0802, Fax +90 212 343 0842

www.ivoclarvivadent.com

Ivoclar Vivadent Limited

Compass Building, Feldspar Close, Warrens Business Park
Enderby, Leicester LE19 4SD, United Kingdom
Tel. +44 116 284 7880, Fax +44 116 284 7881

www.ivoclarvivadent.co.uk

Ivoclar Vivadent, Inc.

175 Pineview Drive, Amherst, N.Y. 14228, USA
Tel. +1 800 533 6825, Fax +1 716 691 2285

www.ivoclarvivadent.us