# IPS® Object Fix Putty / Flow

## Instructions for Use

## Gebrauchsinformation

## Mode d'emploi

## Istruzioni d'uso

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## Bruksanvisning

- Auxiliary firing paste
- Brennhilfspaste
- Pâte réfractaire support de cuisson
- Pasta di cottura ausiliaria
- Pasta auxiliar de cocción
- Pasta auxiliar de queima
- Brännpasta

For dental use only.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a licensed dentist.

Made in Liechtenstein  
Ivoclar Vivadent AG,  
FL-9494 Schaan / Liechtenstein

607635/0910/Rev 1

ivoclar  
vivadent®  
technical

## English

### Description

IPS Object Fix Putty and IPS Object Fix Flow are auxiliary firing pastes to support all-ceramic restorations during the firing and/or crystallization procedure. The pastes are used to stabilize and secure the corresponding restoration on the firing tray. The pastes are distinguished by their viscosities and can be used according to personal preference. Given their consistency, IPS Object Fix Putty / Flow are easy to apply, as well as easy to remove after the firing procedure.

### Delivery forms

- IPS Object Fix Putty – Syringe
- IPS Object Fix Putty – Single Dose
- IPS Object Fix Flow – Syringe

### Indication

- Stabilization of IPS e.max® CAD restorations during crystallization directly on the IPS e.max CAD Crystallization Tray or on the IPS e.max CAD Crystallization Pins.
- Securing of IPS Empress® restorations on metal pins during the firing procedure.

### Contraindication

- IPS Object Fix Putty / Flow cannot be used for the sintering procedure of zirconium oxide frameworks.

### Composition

Oxides, water, thickening agent

### General note

The failure to observe the stipulated application steps may lead to failure.

### Application

*IPS e.max CAD HT/LT, fully anatomical (partial crowns, crowns)*

1. Before crystallization, always clean the restorations with ultrasound in a water bath and/or steam jet. Do not blast with Al<sub>2</sub>O<sub>3</sub> or polishing beads.
2. Select the largest possible IPS e.max CAD Crystallization Pin (S, M, L) that best „fills“ the inside of the restoration.
3. Fill the inside of crowns with IPS Object Fix Putty / Flow up to the restoration margin.
4. Close syringes immediately after use to prevent the pastes from drying out.
5. Press the selected IPS e.max CAD Crystallization Pin deeply into the Putty / Flow material so that it is adequately secured.

- Smooth out displaced Putty / Flow using a plastic spatula (e.g. OptraSculpt®) so that the pin is securely in place and the restoration margins optimally supported.
- Clean off any possible residue adhering to the outer surface of the restoration with a brush or cotton swab dampened with water and dry.
- Depending on the clinical situation, characterize and glaze the restoration using IPS e.max CAD Crystall./Shades, Stains, and Glaze.
- For the combination firing (crystallization and glaze firing), place the restoration on the IPS e.max CAD Crystallization Tray and fire using the stipulated parameters.
- After crystallization, allow the restoration to cool to room temperature and remove it from the hardened IPS Object Fix Putty/ Flow.
- Remove any adhering residue with ultrasound in a water bath or steam jet. Do not blast with Al<sub>2</sub>O<sub>3</sub> or polishing beads.

*IPS e.max CAD HTILT, partially reduced, IPS e.max CAD MO framework*

- Completely fill the inside of the crowns with IPS Object Fix Putty / Flow prior to crystallization and even allow some excess.
- Close syringes immediately after use to prevent the pastes from drying out.
- Place the restorations filled and supported with IPS Object Fix Putty / Flow on the IPS e.max CAD Crystallization Tray. Do not use any other firing trays.
- After crystallization, allow the restoration to cool to room temperature and remove it from the hardened IPS Object Fix Putty/ Flow.
- Remove any adhering residue with ultrasound in a water bath or steam jet. Do not blast with Al<sub>2</sub>O<sub>3</sub> or polishing beads.

*IPS Empress® Esthetic, IPS Empress CAD*

- Remove the characterized and/or veneered restorations from the die.
- Apply a small amount of IPS Object Fix Putty / Flow to the inner side of the restoration and secure the restoration on the firing pin (metal) of the honey-combed firing tray.
- As an alternative, IPS Object Fix Putty / Flow can also be applied on the metal pin and the restoration placed on the pin.
- Close syringes immediately after use to prevent the pastes from drying out.
- After the firing procedure, allow the restoration to cool to room temperature and remove it from the hardened IPS Object Fix Putty/ Flow.

- Remove any adhering residue with ultrasound in a water bath or steam jet. Do not blast with Al<sub>2</sub>O<sub>3</sub> or polishing beads.
- Subsequently, proceed according to the corresponding Instructions for Use.

### Storage

- Protect from direct sunlight and temperatures over 30 °C / 86 °F.
- Close syringes immediately after use.
- Remove IPS Object Fix Putty / Flow syringes only from the aluminium bag immediately before the first application.
- IPS Object Fix Putty Single Dose can be stored at room temperature.
- Keep out of the reach of children.
- Do not use the product after the indicated date of expiration.

### Storage tip IPS Object Fix Putty/Flow Syringe

**How can IPS Object Fix Putty or Flow be prevented from drying out in rooms with a relatively low humidity of less than 40% (particularly during the winter season)?**

The water-containing IPS Object Fix Putty and Flow pastes are packed into the 100% tight aluminium bag. The air in the bag is saturated with moisture. Consequently, no further moisture can escape from the syringe.

In order to avoid a moisture loss in the material even after dispensing IPS Object Fix Putty or Flow from the syringe

- close the syringe **immediately** after use
- store the syringe – after removal from the aluminium bag – in a **resealable plastic bag (e.g. Minigrip) or container with a moist atmosphere**. For this purpose, add a little water to the bag or container or wrap the syringe into a moist paper tissue.

**Date information prepared:** 09/2010 Rev. 1

These materials have been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the materials for their suitability and use for any purpose not explicitly stated in the Instructions. Description and data constitute no warranty of attributes.

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## **Ivoclar Vivadent Limited**
## **Ivoclar Vivadent, Inc.**