Heliosit®

Orthodontic

| | |
|-----------------------------|--------------------------|
| Instructions for Use | Bruksanvisning |
| Gebrauchsinformation | Brugsanvisning |
| Mode d'emploi | Käyttöohjeet |
| Istruzioni d'uso | Bruksanvisning |
| Instrucciones de uso | Productinformatie |
| Instruções de Uso | Οδηγίες Χρήσεως |

- Light-curing bonding material for brackets
- Lichthärtendes Befestigungsmaterial für Brackets
- Matériau de scellement photopolymérisable pour les brackets
- Materiale composito fotoindurente per il fissaggio di bracket
- Material fotopolimerizable para la fijación de brackets
- Adesivo fotopolimerizável para fixar brackets
- Ljushärdande bonding material för brackets
- Lyspolymeriserende cement til brackets
- Valokovetteinen sidosmateriaali käytettäväksi braketiteihin
- Lichtuithardend bevestigingsmateriaal voor brackets
- Φωτοπολυμεριζόμενο, μονοφασικό υλικό, με υψηλή φωτοδιαπερατότητα για τη συγκόλληση ορθοδοντικών αγκυλίων (brackets).

For dental use only!

Caution: U.S. Federal Law restricts this device to sale by or on the order of a licensed dentist.

Made in Liechtenstein
Ivoclar Vivadent AG, FL-9494 Schaan

CE 0123

ivoclar
vivadent®
clinical

557074/07032/WE3/G

English

Instructions for Use

Description

Light-curing, highly translucent single-component bonding material for brackets in orthodontia.

Composition

The monomer matrix consists of urethane dimethacrylate, Bis-GMA and decandiol dimethacrylate (85 wt%). The filler consists of highly dispersed silicon dioxide (14 wt%). Additional contents are catalysts and stabilizers (1 wt%).

Indication

Heliosit Orthodontic is used for the bonding of brackets.

Contraindication

If the patient is known to be allergic to any of the ingredients in Heliosit Orthodontic, the material should not be used.

Side effects

Systemic side effects are not known to date. In individual cases, allergic reactions to composite materials have occurred.

Interactions

Not known to date.

Application

1. Clean enamel surface thoroughly.
2. Isolate with cotton rolls and blown air.
3. Etch enamel surfaces (e.g. with Total Etch for 30 to 60 seconds). Heavily fluoridated enamel may have to be etched longer.

Note: The etched enamel surfaces should be of a white chalky appearance. If this is not the case (e.g. because of contamination with saliva), etch again for another 30 seconds.

- Rinse and dry the etched enamel surfaces thoroughly.
- Apply Heliolit Orthodontic to the undersurface of the metal bracket cleaned according to the instructions of the manufacturer.

Note: Prepare ceramic brackets according to the instructions of the manufacturer.

- Position the bracket on the tooth surfaces. Cure with a halogen light (e.g. Astralis) from the cervical for 20 seconds to fix the bracket in place. For complete polymerization, cure for another 20 seconds from the incisal.

After the second polymerization phase, the bond strength of the material is high enough to allow the removal of excess and the exertion of immediate stress required for correcting the positioning of teeth.

Warning

Avoid contact of uncured material with the skin/mucous membrane or eyes. If uncured, Heliolit Orthodontic may have a slightly irritating effect and cause a sensitizing reaction to methacrylates.

Notes

- Reseal Heliolit Orthodontic syringes immediately after use.
- Exposure to light may cause premature polymerization.
- Brackets must be prepared (cleaned) according to the instructions of the bracket manufacturer.

Storage

- Store at 2–28 °C / 36–82 °F.
- Shelf life: see date of expiration
- Do not use Heliolit Orthodontic after the indicated date of expiration.

Keep the material out of the reach of children.

For use in dentistry only.

Date information prepared

07/2003

Manufacturer

Ivoclar Vivadent AG

FL-9494 Schaan/Liechtenstein

This material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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Ivoclar Vivadent S.A.
Ivoclar Vivadent AB
Ivoclar Vivadent, Inc.